



DuraBase

CHAIRSIDE SOFT CUSHION REBASE

- Self Curing
- Cures in the Mouth
without Patient discomfort
- Long lasting for prolonged
adjustment periods
- Easy fabrication of
soft tissue models
- High color stability
- Economical and easy to use



*Since 1894: Over 100 years
of Reliable Service to the Dental Community*

INSTRUCTIONS:

1. Clean and roughen denture area to be rebased. Dry denture completely.
2. Lubricate the labial and buccal surfaces within $\frac{1}{8}$ inch of the peripheral border.
3. Paint reline area with DuraBase Repair Liquid. This assures a chemical bond.
4. Recommended powder/liquid proportions:
by volume
Powder 10ml —large plastic vial
Liquid 5ml —small plastic vial
Add liquid to powder in mixing cup and stir thoroughly for only 30 seconds.
5. Wait approximately 1 minute and apply mixture to denture.
6. Insert denture in mouth and have patient close mouth lightly into occlusion.
7. After approximately 3-4 minutes in the mouth, muscle-trim the borders, then remove the denture and trim the excess. Rinse the denture in cold tap water and reseat in patient's mouth.
8. DuraBase Soft sets in approximately 15 minutes when cured in the mouth. Remove denture after curing, trim any excess material with a sharp blade or scissors. To smooth edges, a hot spatula can be used.

RICHARD L. MILANO VIA TIEPOLO, 10126 TORINO, ITALY



Dental Mfg. Co.

P.O. Box 38
Worth, IL 60482

(708) 597-6694 FAX (708) 597-7560