ENGLISH

General Information
3M™ ESPE™ Adper™ Scotchbond™ 1 XT Adhesive is a simple, moist bonding adhesive containing 10% , 5M colloidal ltril.

Adper Scotchbond 1 XT adhesive offers the dental practitioner a wide range of applications. These include bonding to all classes of direct composite restorations as well as procedures involving porcelain, composite, metal repair, set amalgam, root surface desensitization and bonding of porcelain veneers with 3M™ ESPE™ RelyX™ Veneer Cement System and RelyX™ Ceramic Primer.

After light curing Adper Scotchbond 1 XT adhesive, it may also be used as amalgam and indirect bonding procedures when combined with 3M™ ESPE™ RelyX™ ARC Adhesive cement bonding procedures. It may also be used for direct bonding procedures or as a low film thickness (approximately 10µm) of cured Adper Scotchbond 1 XT.

Adper Scotchbond 1 XT adhesive is available in two delivery systems, a unit dose delivery and a multi-use vial dispenser.

Use of etchant is crucial for both enamel and dentin surfaces.

Indications

- All direct, light-cured bonding procedures.
- Indirect bonding procedures when combined with RelyX ARC Adhesive Resin Cement.
- Root surface desensitization.
- Porcelain and composite repair.
- Bonding veneers in combination with RelyX Veneer Cement.

Precautionary Information for Patients and Dental Personnel
3M™ ESPE™ Scotchbond™ Universal-Adhäsiv enthält 32% Phosphorsäure. Acid by weight. Contact with eyes or skin may cause burns. Wear suitable protective clothing, gloves and eyeface protection. In case of contact, flush area with plenty of water and seek medical advice.

Precautionary Information for Patients:

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

Precautionary Information for Dental Personnel:

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate any product contacts gloves, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed. 3M ESPE MSDS can be obtained from www.3MESPE.com or contact your local subsidiary.

Hazardous Information

This product is a flammable material.

Recommendations
Use 3M™ ESPE™ Vitrebond™ Light Cure Glass Ionomer Liner/Base in areas of resin infiltration and 1 restorations. If pulp exposure has occurred, use a minimum amount of calcium hydroxide followed by an application of Vitrebond liner/base. Adper Scotchbond 1 XT adhesive will bond to Vitrebond liner/base whether or not the adhesive was treated with etchant.

Adper Scotchbond 1 XT adhesive includes an etch of enamel and dentin as a part of the product application. It is intended that the surfaces be left moist after rinsing. Excess surface moisture should be removed by blotting.

Adper Scotchbond 1 XT adhesive is cured by exposure to visible light. The light curing times instructed with this product assumes the use of a 3M ESPE light curing unit or other dental visible curing light of comparable intensity. Curing lights should be tested for proper output using a reliable light metering system.

Air used for drying should be free of oil and water contaminants.

Sensitivity

Some patients may experience transitory postoperative sensitivity. The risk of sensitivity can be minimized by the following measures:

- Tooth Preparation
 - Remove minimal tooth structure.
 - Use proper isolation. Use of a rubber dam is highly recommended. Use adequate pulp protection. Use a glass ionomer or resin-modified glass ionomer liner/base (3M™ ESPE™ Vitrebond) in areas of deep cavitation.
- Adhesive Application
 - Use of compressed air is not recommended to remove pooled water remaining after the etch step—Blot excess moisture from the preparation using a cotton pellet or mini- sponge.
 - Apply adhesive immediately after blotting.
 - Place restorative material in increments, curing each increment separately. Adequately cure restorative according to instructions for shade and thickness requirements and light cure.
 - Adjust occlusion carefully. Check for hyperocclusion, particularly in lateral occlusion contacts.

Etchant Syringe Assembly

- Protective eyewear for patients and staff is recommended when using the delivery system.

- Prepare the delivery system: Remove cap from etchant syringe and SAVE. Twist a blue disposable tip securely onto the syringe. Holding the syringe with the tip away from you, apply a dental sealant, express a small amount of etchant onto a dispensing pad or a 2x2 gauge to assure that the delivery system is not clogged.

If clogged, remove the dispensing tip and express a small amount of etchant directly from the syringe. Remove any visible plug, if present, from the syringe opening. Replace dispensing tip and again express etchant. If clog remains, discard dispensing tip and replace with a new one. Bend the dispensing tip to a desired angle. Place bend midway along tip. Do not bend dispensing tip at its hub as this may cause the tip to break free.

- Delivery system storage: Remove used dispensing tip and discard. Twist on storage cap. **Storage of the etchant syringe and dispenser:** Store the etchant syringe and storage cap **will allow drying of the etchant and consequent clogging of the storage.** Replace storage cap with a new dispensing tip at next use.

- If desired, the etchant may be extruded onto a dispensing pad and applied with a brush or other appropriate instrument.
- If a liquid etchant is desired, the etchant may be dispensed into a dappen dish and stirred to increase its viscosity.
- Disinfectant: Do not use dispensing tip. Replace syringe cap. See Storage and Use section for disinfection.

Instructions for Use

Dispensing Adper Scotchbond 1 XT Adhesive Unit Dose:

Attention: To minimize risk of accidental contact with eyes and skin, hold the shaft of the disposable applicator above the opening where the applicator enters the foil package with your thumb and index finger. Do not activate the etchant with your index finger. Wear your thumb and index finger over the other hand, squeeze the large blister to transfer the adhesive into the chamber enclosing the applicator. Briefly spin the applicator to fully saturate with adhesive.

Vial:

Pinch the sides of the cap to release the locking mechanism and flip the cap back to remove the dispensing tip. Squeeze out the exact number of drops you need into the dispensing tip. When finished, flip the cap back until it is secured by its locking mechanism.

Directions

- Instructions for direct light cure restorations in enamel and dentin**
 - Isolation: Rubber dam is the preferred method of isolation.

- Cavily preparation: Prepare cavity with minimal tooth reduction. Bevel cavosurface enamel margins.

- Etching: Apply etchant to enamel and dentin. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water.
- Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel, dentin and any residual restorative material. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water.

Storage and Use

This product is designed to be used at room temperature. If stored in cooler, allow product to reach room temperature prior to use. Shelf life of the unit dose at room temperature is 24 months. Shelf life of the vial delivery at room temperature is 36 months. Ambient temperatures routinely higher than 27°C (80°F) may reduce shelf life. See outer package for expiration date.

- Adper Scotchbond 1 XT adhesive should be capped immediately after use to minimize evaporation.
- Do not expose materials to elevated temperature or intense light.

Disinfect this product using an intermediate level disinfection process (liquid contact) as recommended by the Centers for Disease Control and endorsed by the American Dental Association. Disinfectant: Rinse and soak in disinfectant solution in a 3M™ ESPE™ MMWR, December 19, 2003;SD-RR-17). Centers for Disease Control and Prevention.

- Etching: Apply etchant to both enamel and dentin. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water. An additional 15 second etch time may be appropriate for teeth that were not prepared using a diamond or bur. Residual organic matter can also require additional etch time.

- Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. **Do not** over cure.

Adhesive application to veneer: Apply 1 coat of adhesive to the acid etched, silane treated veneer. Dry thoroughly. Do not light cure.

- Luting material application to veneer: Apply RelyX Veneer Cement to the bonding surface of the veneer.

Sealing and curing: Carefully seat the veneer. Clean excess luting cement from the veneer margins. Cure each area of the veneer for times recommended by the luting cement manufacturer. We recommend curing the gingival margin first, followed by the body and the incisal margin. Avoid direct contact with the light-guide during initial curing.

Instructions for bonding to composite and set amalgam

- Isolate the rubber dam and remove all isolation.
- Roughen the existing material. Roughen the surface of existing amalgam or composite using either a bur, diamond or a sandblast technique.
- Etching: Apply etchant to enamel, dentin and existing restorative material. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini sponge. The surface should appear glistening without pooling of water.
- Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel, dentin and any residual restorative material. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini sponge. The surface should appear glistening without pooling of water.
- Masking: In the case of set amalgam, mask the metal surface with an appropriate masking agent. Light cure according to the manufacturer’s instructions for use.
- Restorative placement: Refer to manufacturer’s instructions for placement, cure and finishing of restorative material.

Instructions for porcelain repair

- Isolate the rubber dam. Blot the surface with the preferred method of isolation.
- Preparation: Clean the surface to be repaired with a slurry of plain flour of pumice. Rinse and dry thoroughly. Roughen the surface of existing metal or porcelain using either a bur, diamond or a sandblast technique. Be careful to remove all loose porcelain and bevel the margin. Remove surface glaze 1mm beyond the margin.
- Etching: Apply etchant to all substrates. Wait for 15 seconds. Rinse for 10 seconds. Dry 5 seconds.
- Slane treatment for porcelain and metal: Apply RelyX Ceramic Primer (No. Z721) to the etched surface and dry.

- Adhesive: Apply 2 consecutive coats of Adper Scotchbond 1 XT adhesive to silane treated porcelain or metal. Dry gently for 5 seconds. Light cure for 10 seconds.
- Masking: To opacity the metal before the final composite placement, mask the metal surface with an appropriate masking agent. Light cure according to the manufacturer’s instructions for use.
- Restorative placement, cure and finishing: Refer to manufacturer’s instructions for placement, cure and finishing of restorative material.

Instructions for root surface desensitization

- Lightly clean the root surface with a plain pumice. Rinse and blot dry.
- Etching: Apply etchant to the etched dentin. Wait for 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water.
- Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. Light cure for 10 seconds.
- Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin. Dry gently for 5 seconds. Light cure for 10 seconds.
- Remove the oxygen inhibited layer with a moistened gauze.

Instructions for Amalgam and Indirect Bonding Procedures (using RelyX ARC adhesive resin cement)

Physical properties of today’s esthetic indirect restorations require that they be bonded into place to maximize the strength of the restoration as well as the tooth. A general principle is that the light cure adhesives used on the restoration should be applied in a thick layer. Although indirect direct adhesives have a higher film thickness and cannot be used under a fixed prosthesis. However, Adper Scotchbond 1XT adhesive is ethanol/water based, has a low film thickness (approximately 10µm) and should not interfere with the setting of indirect restorations.

Note: Care is required with any bonding agent used beneath precision castings because even a thin thickness may preclude successful casting. **Avoid adhesive pooling in areas of the preparation that would effect the fit of any prosthetic device.**

Instructions for bonding crowns, bridges (including resin-bonded bridges), inlays and onlays

- Remove temporary restoration. Trial-fit the final restoration with light finger pressure to evaluate the fit, shade and marginal integrity. Adjust if necessary.
- Prepare the bonding surface of the indirect restoration and the core build up, if applicable. The bonding surface should have been etched with hydrofluoric acid by the dental laboratory. Metal and amalgam bonding surfaces should be roughened, preferably using an air abrasion system, diamond or bur. Any composite surfaces should be roughened with a diamond, bur or air abrasion system. Glass ionomer build-ups should be pumiced with a slurry of plain flour of pumice.
- Slane treatment (porcelain or porcelain/metal indirect restorations): Apply RelyX Ceramic Primer to the bonding surface of the indirect restoration. Dry for 5 seconds.
- Clean the prepared teeth in preparation for seating and bonding using a plain flour of pumice slurry. Rinse and dry thoroughly, isolate from moisture and adjacent teeth.
- Etching: Apply etchant to both enamel and dentin. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water.
- Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents being careful to avoid excess adhesive on all prepared substrates.
- Light cure preparation for 10 seconds per bonding surface.
- Dispense the appropriate amount of cement onto a mixing pad and mix for 10 seconds.
- Apply and evenly distribute a thin layer of cement to the bonding surface of the indirect restoration.
- Slowly seat and hold restoration in proper occlusion. Begin clean-up of excess cement approximately 3-5 minutes after seating. **Optionl: If excess cement is removed immediately after seating, each cement surface/margin will be light cured for 40 seconds.**
- Once the restoration is seated, each cement surface/margin may be light cured for 40 seconds or allowed to self-cure. Note for porcelain and pre-cured composite restorations, each cement surface/margin will be light cured for 40 seconds.
- Instruct patient to avoid applying any pressure for 10-15 minutes.

Instructions for bonding endodontic posts

- Prepare the endodontically treated tooth to receive the post (a root apex sealer and gutta percha filling approximately one third of the root canal are recommended). **Trial fit and adjust post as needed.** Bond to cast posts can be enhanced by using an air abrasion system and then applying RelyX Ceramic Primer. Dry for 5 seconds.
- Etching: Apply etchant to the prepared tooth. Wait 15 seconds. Rinse for 10 seconds. Dry for 2 seconds. Remove excess moisture with an absorbent paper point.
- Adhesive: Apply a uniform coat to etched enamel and dentin. Remove excess pooled adhesive with absorbent paper point. Air thin for five seconds to evaporate solvents.
- Light cure for 10 seconds. (A light transmitting post of appropriate size may be used for additional curing).
- Dispense the appropriate amount of cement onto a mixing pad and mix for 10 seconds.
- Apply cement to the bonding surface. The preparation (in and around canal using a liquid endodontic probe). Place a thin layer of mixed cement on post.
- Seat the post. While holding in place remove excess cement. Light cure for 40 seconds from the occlusal surface to allow immediate placement of core build-up material.

Instructions for bonding amalgam to tooth structure

- Isolation: Rubber dam is the recommended method of isolation.
- Cavily preparation: Prepare a standard amalgam cavily preparation. Roughen residual restorative materials with an air abrasion system or a bur.
- Mainn application: Lightly lubricate the inner surface of the matrix band with wax or petroleum jelly before placement.
- Etching: Apply etchant to enamel, dentin and any residual restorative. Wait for 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini sponge. The surface should appear glistening without pooling of water.
- Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel, dentin and any residual restorative material. Wait 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents.
- Light cure for 10 seconds.
- Dispense the appropriate amount of cement onto a mixing pad and mix for 10 seconds.
- Use a brush or appropriate applicator to place cement in adhesive-sealed preparation. **Triturate amalgam during placement of cement.**
- Condense and burish amalgam in the usual way.
- Instruct patient to avoid applying any pressure for 10-15 minutes.

Storage and Use

- This product is designed to be used at room temperature. If stored in cooler, allow product to reach room temperature prior to use. Shelf life of the unit dose at room temperature is 24 months. Shelf life of the vial delivery at room temperature is 36 months. Ambient temperatures routinely higher than 27°C (80°F) may reduce shelf life. See outer package for expiration date.
 - Adper Scotchbond 1 XT adhesive should be capped immediately after use to minimize evaporation.
 - Do not expose materials to elevated temperature or intense light.
- Disinfect this product using an intermediate level disinfection process (liquid contact) as recommended by the Centers for Disease Control and endorsed by the American Dental Association. Disinfectant:** Rinse and soak in disinfectant solution in a 3M™ ESPE™ MMWR, December 19, 2003;SD-RR-17). Centers for Disease Control and Prevention.

Diisposal

See the Material Safety Data Sheet (available at www.3MESPE.com or through your local subsidiary) for disposal information.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty
3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability
Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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