ENTRETIEN ET NETTOYAGE:

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INSTRUCTIONS FOR USE**USER'S INFORMATION**

Personal Protective Equipment (Eye Protectors)
EN166

MODELS:

Goggles : Ventor, Dusty
Spectacles : Namib

These protective spectacles and goggles supplied by Artelli have been assessed to EN166 : 2001 by Inspec International Ltd., Upper Wingbury Courtyard, Wingrave, Aylesbury, Bucks, HP22 4LW, United Kingdom. Notified Body No: 0194. They do not provide an unlimited protection for eyes and you should read the instructions completely before using them.

LIQUID SPLASH RESISTANT GOGGLES: VENTOR

These goggles (Ventor) have been designed for protection against high speed, medium energy impact particles and droplets of liquids.

**IMPACT RESISTANT GOGGLES: DUSTY
IMPACT RESISTANT SPECTACLES: NAMIB**

These goggles (Dusty) and spectacles (Namib) have been designed for protection against high speed, low / medium energy impact particles.

They are very resistant but not unbreakable. It is the responsibility of the user to understand the limits of their usage. Additionally you should check with your supervisor of employer before using this product to determine whether or not is suitable for your job and in your area.

CARE AND CLEANING:

Inspect regularly and immediately replace when damaged. Carry in the box, bag or case provided and not unprotected in a packet with other items. When not in use, store in original packaging at normal room temperature if possible. Clean with soap and lukewarm water, rinse then dry with a clean, lint free cloth or tissue. Disinfect when required with mild household detergent or medical disinfectant. Do not use solvents.

PACKAGING:

The use and life of the product is dependent on the care and condition of use. When transporting the product, use the original packaging where possible or a similar alternative. Items in transport should not be carried loose with other items such as tools or other metal objects. None of the materials used in the manufacture of this device are known to cause allergic reactions to persons of normal skin type. However persons of susceptible skin type should be aware that materials which contact the skin, may in some cases cause allergic reactions.

SIGNIFICANCE OF THE MARKING ON THE FRAME AND ON THE OCULAR:

EN166 : Standard number for Personal Eye Protection - Specifications

ART : Identification of the manufacturer

O : Symbol for original ocular

7 : Symbol for replacement ocular (optional)

If the ocular and frame do not carry identical markings relating to mechanical strength the whole eye protector is classified as the lower.

Product marking	Mechanical strength
S	Increased robustness
F	Low energy impact
B	Medium energy impact
A	High energy impact
	Optical performance
1	Optical class 1
2	Optical class 2
3	Optical class 3 (not suitable for long periods of use)
K	Ocular resistant to damage by fine particles (optional requirement)
N	Ocular resistant to fogging (optional requirement)
	Field(s) of use
3	Droplets and splashes of liquids
4	Large dust particles
5	Gas and fine dust particles
8	Short circuit electric arc
9	Molten metals and hot solids

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Testinstitut: Inspec International Ltd.