VOCO

Calcicur

****CE**⁰⁴⁸²**

****GB****
Instructions for use

Compositon:
Radiopaque water-based calcium hydroxide paste.

Indications:
– indirect capping for the treatment of caries profunda
– direct capping when the pulp is opened or a pulpotomy is performed
– lining of cavities to prevent exposure to acid media when using cements
– for temporary filling of root canals if root growth has not terminated yet

Directions:

Clean and dry the cavity before applying **Calcicur** paste in thin layers until the desired thickness is reached. Allow each layer to dry briefly or dry carefully with an air syringe. Do not apply paste to the cavity margins or remove material adhering to them. Then insert lining and filling.
For root canal fillings, first prepare and clean the root canal (hydrogen peroxide 3% and sodium hypochlorite solution) and insert paste into the root canal with a suitable instrument (Lentulo). Avoid overfilling (apical) and air pockets; then insert the filling. Checking the root canal filling with an x-ray is recommended. Renew the root canal filling after longer periods of time (more than 6 - 8 months).

Supplementary information/precautionary measures:
Improper storage can cause drying or separating of the paste.

If the paste material in the protective capsule has thickened, the protective function can be restored by adding 1 - 2 drops of distilled water.

Calcium hydroxide is caustic to skin and mucous membranes. Avoid contact with the eyes!
Do not treat those areas of the hard tooth substance with calcium hydroxide, which serve for bonding to dental materials! Do not apply **Calcicur** in thick layers, if caries is remaining in the dentine, or as definite filling or luting material.

Storage:
Reattach the protective cap to syringe immediately after use. Clean and sterilise cannula after direct application. Do not use hardened paste.
Store at temperatures between 15°C - 23°C. Do not use **Calcicur** after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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