NIGHT CLEANER

INSTRUCTIONS FOR USE

"I FEEL GOOD"

EMS⁺

SUMMARY

EN ENGLISH p.2

DE DEUTSCH p.2

FR FRANÇAIS p.2

IT ITALIANO p.2

ES ESPAÑOL p.3

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NO NORSK p.3

SE SVENSKA p.3

FI SUOMI p.4

NL NEDERLANDS p.4

DK DANSK p.4

PL POLSKI p.4

CZ ČESKÝ p.5

SI SLOVENSKO p.5

SK SLOVENSKY p.5

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ID INDONESIA p.7

MS BAHASA MELAYU p.8

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NIGHT CLEANER ensures the decontamination and prevents biofilm formation in water lines of all EMS AIRFLOW® units and extends their life when used daily.

NIGHT CLEANER is a ready-for-use solution to be used daily in the blue NIGHT CLEANER bottle (EG-120).

Scope of activity:
- Bactericidal/ fungicidal
- Removes and prevents lime and algae formation

Before Use

Before using this product, please carefully read and follow these Instructions for Use and the Instruction for Use of the medical device intended to be processed. Pay special attention to the safety precautions and warnings.

Incident Notification

Any serious incident that has occurred in relation to the **NIGHT CLEANER** should be reported to the manufacturer and the competent authority of the country where the user and/or patient is established.

Intended Use and User

NIGHT CLEANER ensures the decontamination and prevents biofilm formation in water lines of all EMS AIRFLOW® units and extends their life when used daily. **NIGHT CLEANER** removes and protects from algae and limescale, after longer idle times or heating of the process water.

NIGHT CLEANER is a medical device and shall be used only by qualified dentists and dental hygienists.

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Contient de la phénylalanine.

Application:

Before starting the application (overnight, weekend or holidays), fill the NIGHT CLEANER bottle with **NIGHT CLEANER**. **Remove handpieces and instruments from the target device.**

Operate overnight cleaning according to the operating instructions of your AIRFLOW® device and fill the waterlines with **NIGHT CLEANER**.

Afterwards, turn off the treatment unit. Allow **NIGHT CLEANER** to remain in the lines (minimum 12 hours) until the practice starts to work again.

When the treatment unit is put into operation again, start the rinsing process according to the operation instructions of your AIRFLOW® device.

Do not reuse.

For further advice, please consult the instructions of your AIRFLOW® device.

Transport and storage conditions:

Transport temperature: from -20°C to 40°C
Storage temperature: from +5°C to 30°C

Disposal of the product

Product residues must be disposed of as non-hazardous waste in compliance with the Directive 2008/98/EC on waste as well as national and regional regulations. Do not mix with other waste materials.

Disposal of the packaging

Contaminated packaging must be emptied optimally and can be recycled.

0123 REF DV-154

Content:

e 800 ml

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

Contains a source of phenylalanine.

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E.M.S. Electro Medical Systems S.A.