CERAM-ETCH

9% BUFFERED HYDROFLUORIC ACID GEL

ITENA®

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EN CERAM-ETCH is a viscous hydrofluoric acid gel indicated for glass-based porcelain and ceramics etching.

Indications: CERAM-ETCH is indicated for the intrados preparation of: vitreous ceramics crowns, veneers and inlays before cementation with resin type composites. It is also indicated for all-ceramic and metal-ceramic fractured crown repair. For an optimal preparation, it is advised to apply SILAN-IT on the etched surface afterwards.

Instruction:

- 1 - In case of intraoral procedure, isolate the area to be etched with a dam.
- 2 - When preparing fractured porcelain crowns for bonding, abrade the surface of the porcelain with a medium grit bur to remove the glazed surface and to create enough space for a sufficient layer of composite resin. Sandblasting can also be performed. Rinse all debris from the surface with water and dry with moisture and oil free compressed air.

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3 - Apply CERAM-ETCH to the porcelain surface with a micro-brush and leave for 60 seconds.

4 - Rinse the porcelain thoroughly with water to completely neutralize the etching agent and dry well. The etched porcelain should have a frosted white appearance. If not, repeat the etching procedure.

CAUTION: CERAM-ETCH must be used carefully. Avoid contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush with copious amounts of water and seek medical attention.

Composition: Hydrofluoric acid (9%), Water (90%), Thickening agent (1%)

Packaging : 2 syringes (1.2ml)

Storage: Do not store above 24°C. Recap after use and keep refrigerated to optimize freshness of the product.

Expiration date: two years after manufacturing. Do not use after expiry date

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