





# TECHNICAL SHEET

<b>Product</b>	<b>SUPRASPONGES R&amp;S</b>	
<b>Reference</b>		
	Product 50 pieces box	Reference <b>3124553</b>
<b>Picture</b>		
<b>Description</b>	<ul style="list-style-type: none"> <li>• <i>Suprasponges</i> is a surgical haemostatic sponge, manufactured from highly purified first extract grade pork gelatin material for use in various surgical procedures where traditional haemostatic is difficult and use of other non-absorbable materials is undesirable.</li> <li>• <i>Suprasponges</i> can be effectively used in dental Surgery. <i>Suprasponges</i> may be used in oral and maxillary surgery, allowing controlling bleeding in extractions and apicentomics especially in patients with a tendency to bleed extensively. <i>Suprasponges</i> can also be used to achieve immediate haemostasis in: <ul style="list-style-type: none"> <li>- Dentoalveolar surgery and extraction of teeth in patients with risk of haemorrhage.</li> <li>- Surgical removal of oral tumors.</li> <li>- Surgical treatment of periapical abscesses.</li> <li>- Maxillo-facial surgery.</li> </ul> </li> </ul>	
<b>Characteristics &amp; Technical data</b>	<ul style="list-style-type: none"> <li>• <i>Suprasponges</i> is a non-toxic, non-allergenic, non-immunogenic, non-pyrogenic and biocompatible product.</li> <li>• <i>Suprasponges</i> approximately 40 to 50 times, its weight of water/blood and adheres easily to the bleeding site. The uniform porosity of <i>Suprasponges</i> guarantees a favorable haemostatic.</li> <li>• When implanted in vivo, it is completely absorbed within 3 to 4 weeks</li> <li>• <i>Suprasponges</i> have a porous structure which activates the thrombocytes at the moment blood comes in contact with the matrix of the sponge. This causes the thrombocytes to release a series of substances which promote their aggregation, thus enabling them to act as a catalyst for the formation of the fibrin</li> </ul>	
<b>Storage</b>	<ul style="list-style-type: none"> <li>• <i>Suprasponges</i> is available in bulk packing. Once the package is opened, contents are subject to contamination. It is recommended that <i>Suprasponges</i> be used as soon as the package is opened and unused contents discarded. It should not be reused or re-sterilized as it may lose its physical characteristics as well as sterility.</li> <li>• <b>Storage:</b> The product should be protected from direct sunlight and heat, and stored in its original packaging in a clean, dry room at a temperature from 5°C to 30°C</li> </ul>	

<b>Guarantee</b>	5 years
<b>Indication of use</b>	
<b>Legal Information</b>	<p>Medical device class III. CE marking 2460.  Notify body DNV GL Nemko presafe AS, Veritasveien 3, 1363 Hovik, Norway.  European representative : Wellkang Ltd. Suite B, 29 Harley street, LONDON W1G 9QR,  England, UNITED KINGDOM.</p>

Version 4

Updated by RG on the 14/11/2022