

MY
ITENA
CHOICE
#aesthetics

PURE OFFICE

Professional teeth whitening kit



PURE OFFICE

What is best about this kit?

- 35% Hydrogen Peroxide
- Neutral pH: complete removal of the peroxide is easy, with maximum patient comfort
- *A fast and dazzling result*

Express intervention

- Fast procedure: one session only
- Whitening of one devitalized tooth or all teeth


INTRADUCTAL PROTOCOL - Non-vital tooth whitening




1 Take the syringe out of the refrigerator and let it reach room temperature. Protect the tooth to be whitened by setting up an operative field.



2 Evaluate the quality of the root canal filling to prevent any risk of spreading the product.

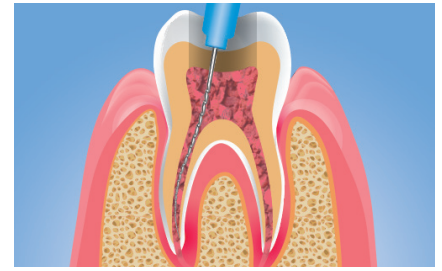


3 Clean the pulp chamber: remove pulp and necrotic tissue residues.

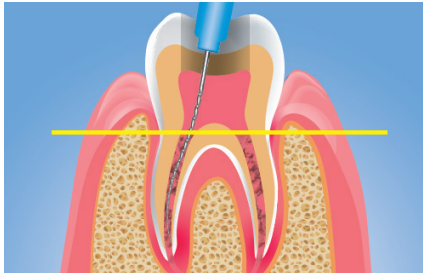

4 The canal should be unblocked approximately 3 mm below the cemento-enamel junction. Isolate the gutta-percha filling by placing a small glass ionomer plug.



5 Place the mixing tip on the PURE OFFICE syringe.

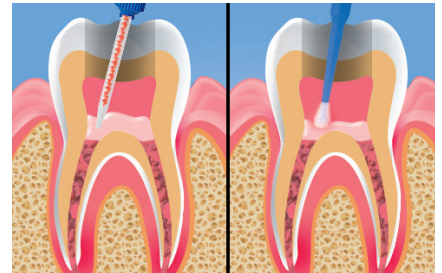

6 Dispense the product into the prepared canal and tamp the product into the cavity with a micro-applicator.



7 Make the temporary closure with a waterproof material.



8 Perform a control after 7 days. Renew the product application if necessary.



9 When the whitening is satisfactory, rinse, dry and place a temporary filling for 5 weeks before final reconstitution.

For which cases?

- Use inside the canal on a devitalised tooth
- External use (for non-EU dentists only);
- The treatment is given in 1-3 stages, in a single session, in the dental office.



Before



During



After

References

PURE OFFICE Intro Kit - 35% Hydrogen Peroxide

EU markets:

1 PURE OFFICE whitening gel syringe (5 g) + 2 PURE PROTECT gingival barrier syringesPROF35-C1

Non-EU markets:

1 PURE OFFICE whitening gel syringe (5 g) + 2 PURE PROTECT gingival barrier syringes PROF35-C1W

PURE OFFICE Value Pack - 35% Hydrogen Peroxide

EU markets:

3 PURE OFFICE whitening gel syringes (5 g) + 6 PURE PROTECT gingival barrier syringes PROF35-RF3

Non-EU markets:

3 PURE OFFICE whitening gel syringes (5 g) + 6 PURE PROTECT gingival barrier syringesPROF35-RW3



Find us on www.itena-clinical.com

Medical Device marked CE0425 for dental care. Only for use by healthcare professionals. Read the instructions in the package leaflet carefully before using the product. Itena Clinical, 31 avenue Georges Clemenceau, 93420 Villepinte - FRANCE - Tel: + 33 1 45 91 61 40 - IND 2 - 06/2021