



DENTO-VIRACTIS SURFACES 79

DENTO-VIRACTIS

Ready to use



Description	Dento-Viractis 79 is recommended for cleaning and disinfecting non-invasive small medical devices (small medical and dental equipment).
Scope of application	For use and small medical equipment.
Characteristics	<ul style="list-style-type: none"> ✓ Wipes particularly large, thick, strong and sweet. ✓ Effective in 1 minute on the H1N1 virus. ✓ Pop-Up System ensuring a faster and a more convenient action.
Commercial presentation	The box of 100 wipes

Antimicrobial activity			Contact time, conditions
Bactericidal	EN 13727		5 min, dirty conditions
	EN 14561		
	EN 14561	MRSA	5 min, dirty conditions
Fungicidal	EN 14562	Aspergillus fumigatus	5 min, dirty conditions
Yeasticidal	EN 13624	Candida albicans	5 min, dirty conditions
	EN 14562	Candida albicans	
Virucidal	EN 14476	HBV – HCV - Adenovirus - Coronavirus - Norovirus – VRS – H1N1 – HSV1	1 min, dirty conditions
		Polyomavirus	1 min, dirty conditions
		Rotavirus	15 min, dirty conditions

ADDITIONAL USES			
Bactéricide	EN 1276	30 sec, dirty conditions	
	EN 13697	5 min, dirty conditions	
Virucide	EN 1650	Candida albicans	5 min, dirty conditions
	EN 1397	Candida albicans	5 min, dirty conditions



DENTO-VIRACTIS SURFACES 79

DENTO-VIRACTIS

Composition	<ul style="list-style-type: none">– less than 5% non-ionic surfactants– fragrance (including limonene)– disinfectants: Ethanol (CAS n° 64-17-5), Alkyl (C12-16) dimethylbenzyl ammonium chloride (CAS n° 68424-85-1), Propan-2-ol (CAS n° 67-63-0) <p>Manufactured in compliance with the NF EN ISO 13485 quality management systems. Data sheet available at www.dentoviractis.com CE marking affixation year: 2003.</p>
Instructions for use	Single use wipes; ready to use. When using for the first time: open the flap, definitively remove the tamper-evident seal, thread the first wipe through the hole, then close the flap. Clean the equipment with the wipe. Leave for between 30 seconds to 15 minutes depending on the desired antimicrobial activity. Allow the surface to air dry. Do not rinse. Close the flap after each use to help prevent the wipes from drying out. Use within 3 months of opening.
Caution	Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents/container at an approved waste disposal facility. Keep out of reach of children. Wear protective gloves. IF IN EYES: rinse cautiously with water for several minutes. Never reuse wipes, they may be less effective and present a risk of cross contamination. Skin tolerance tested under dermatological control.

CE0459 Marking obtained in 2019. Class IIb DM. Version of the sheet: 2.0 of 09/2021

Distribué par :
CADENCE
2 bis, chemin du loup
93290 Tremblay-en-France
FRANCE



Laboratoires Sarbec
10 rue du Vertuquet
59960 Neuville-en-Ferrain
FRANCE
Tel : +33 (0)3 20 69 26 26