




DENTO-VIRACTIS ASPIGERM 95

Dilution:
0,5%

DENTO-VIRACTIS

<p>Description</p>	<p>Dento-Viractis 95 ASPIGERM is recommended for the cleaning and disinfection of aspiration systems, spittoons and amalgam catchers.</p> <div style="text-align: center;">  </div>
<p>Scope of application</p>	<p>For use on surgical and saliva aspiration tubing.</p>
<p>Characteristics</p>	<ul style="list-style-type: none"> ✓ Optimal and non-foaming cleaning ✓ Descaling agent derived from lemon ✓ Fragrance Citrus ✓ Without EDTA, CMR, phosphate
<p>Commercial presentation</p>	<p>1 L bottle with dosing device 5 L bottle</p> <p>Accessories: Pump dispenser of 10mL for 2 L system Pump dispenser of 15 ml for 4 L system</p>

Antimicrobial activity			Concentration	Contact time
Bactericidal	EN 13727	Pseudomonas aeruginosa, Staphylococcus aureus,	0,5%	15 min
	EN 14561	Enterococcus hirae	2%	15 min
Yeasticidal	EN 13624	Candida albicans	0,5%	15 min
Virucidal	Herpes virus simplex type 1 according to EN 14476		0,25%	5 min
	PRV (surrogate of HBV) according to EN 14476		0,5%	1 min



DENTO-VIRACTIS ASPIGERM 95

DENTO-VIRACTIS

Composition	Cationic and non-ionic surfactants, descaling agent, fragrance.
Precautions	Contains D-LIMONENE. May produce an allergic reaction.
Instructions for use	Preparation of a 0.5% strength solution: 1. Dilute 5ml per liter of water (i.e. 20ml in 4l of water). Usage: 1. Circulate the solution through each aspiration tube. 2. Contact time: 15 minutes. 3. Rinse.

Risk phrases:

- R36: Irritating to eyes.
- R50: Very toxic to aquatic organisms.

Security phrases:

- S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- S57: Use appropriate containment to avoid environmental contamination.
- S60: This material and its container must be disposed of as hazardous waste.
- S61: Avoid release to the environment.

Refer to special instructions/safety data sheet.

Labelling:



CE0459 mark obtained in 2019. Class IIa DM. Version of the sheet: 3.0 of 30/09/2021

Distribué par :
CADENCE
2 bis, chemin du loup
93290 Tremblay-en-France
FRANCE



SODEL - ISO 9001/13485
190 rue René Barthélemy, Pôle d'activités
de l'Espérance, 14100 Lisieux, FRANCE.
Tel: +33 (0)2 31 31 10 50
www.sodel-sa.eu - www.alkapharm.fr