



DENTO-VIRACTIS 57

ROTARY INSTRUMENTATION

Ready to use



DENTO-VIRACTIS

Description	Dento- Viractis 57 is a product for the cleaning and disinfection of burs and rotary instrumentation prior to sterilisation.
Scope of application	For use on contaminated reusable burs and rotary instruments.
Characteristics	<ul style="list-style-type: none"> ✓ Dropout protein and mineral stains ✓ Contains an anti-corrosion agent ✓ Product ready for use ✓ Compatible with ultrasonic baths
Commercial Presentation	<p>2 liters bottle ready to use</p> <p>5 liters bottle ready to use</p>

Antimicrobial activity			Contact time
Bactericidal	EN 1040	Pseudomonas aeruginosa, Staphylococcus aureus	5 min
	EN 13727	Pseudomonas aeruginosa, Staphylococcus aureus,	15 min
	EN 14561	Enterococcus hirae	15 min
	NF T 72-190	Pseudomonas aeruginosa, Staphylococcus aureus	5 min
	NF T 72-301	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Escherichia coli	5 min
Fongicidal	EN 1275	Candida albicans,	15 min
	EN 13624	Candida albicans,	15 min
	EN 14562	Aspergillus Niger	15 min
Virucidal	Hepatitis B		10 min
	Herpes virus simplex type 1 according to EN 14476		15 min
	BVDV (surrogate of HCV) according to EN 14476		15 min
	NF T 72-180	Enterovirus polio 1, Adenovirus type 5, Orthopoxvirus of the vaccine	15 min



DENTO-VIRACTIS ROTARY INSTRUMENTATION 57

DENTO-VIRACTIS

Composition	Ethyl alcohol, Potassium Hydroxide, Ethoxylated alcohol.
Precautions	Contains POTASSIUM HYDROXIDE. Contains D-LIMONENE and LINALOOL. May produce an allergic reaction.
Instructions for use	1. Submerge the devices in the Dento-Viractis 57 solution. 2. Immersion time: 15 min. 3. Rinse. The solution must be changed as soon as it becomes dirty or at least once daily.

Risk phrases:

- R10: Flammable.
- R22: Harmful if swallowed.
- R34: Causes burns.

Security phrases:

- S2: Keep out of the reach of children.
- S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- S36/37/39: Wear suitable protective clothing, suitable gloves, and eye face protection.
- S45: In case of accident or if you feel unwell seek medical advice immediately (show the label where possible).
- S60: This material and its container must be disposed of as hazardous waste.

Labelling:



CE Marking obtained in 2008. Version of the sheet: 2.0 of 03/05/2018

CE HYGITECH
0 1 2 0
www.hygitech.com
109 bis avenue Charles de gaulle
92200 Neuilly sur Seine – FRANCE
Tel. +33 (0)1 48 01 32 89