...3TECH

THERMOSEALER

TS-12

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1. Intended use

Thermosealer was developed to seal packages (paper/plastic) for sterilization in steam autoclaves.

2. Box content

- 1 sealer
- 1 instruction manual.

3. Recommendations and warnings

NEVER PERFORM SEALING WITH THE PLASTIC SIDE DOWNWARDS, THIS SEALER MUST ONLY BE USED WITH PAPER/PLASTIC PACKAGES.

- Follow the instructions and guidelines from the sterilization packages manufacturer regarding use instructions, precautions, conservation and storage.
- This equipment works with high temperatures, because of that, avoid contact with hot parts, indicated by a label.
- When packaging instruments, protect the pointy or sharp ones with cotton or gauze to avoid cuts or holes in the packages.
- The amount of instruments must be compatible with the package to avoid ruptures.
- Sealer is able to work on 127V and 220V, as indicated in the identification label, behind the equipment (Fig. 1).

4. Components identification

- 1 On/Off button (side part)
- 2 Heating LED
- 3 Sealing complete LED
- 4 Sealing handle
- 5 Cutting blades holder
- 6 Power cord
- 7 Roll holder
- 8 Identification label (under the equipment)
- 9 Warning sticker

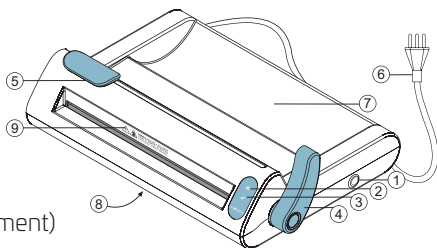
Fig. 1

5. How to use the sealer

- 1 Plug the sealer and press the On/Off button until the beep sounds to turn on the equipment, 3 beeps will sound and the LED will blink, indicating the heating phase. When sealing temperature is reached (three minutes approximately with the handle lifted up, Fig. 1), 2 beeps will sound and the LED will be on, indicating that the sealer is ready to be used.
- 2 Insert the tip of the sterilization package through the sealing entrance, always with the paper side downwards and plastic upwards, pull by the other side and place the roll at the roll holder (Fig. 1). Pull the paper to seal at the desired size.

Never perform sealing with the plastic side downwards in order avoid trouble with access of plastic material on the heater.

- 3 Pull the sealing handle until it locks, a beep will sound (Fig. 2), starting the sealing process (6 seconds). At the ending of this period, the LED ✓ will be on and two beeps will sound continuously.

Note: After 5 minutes with the handle down, the sealer will be automatically turned off.

- 4 Before lifting up the sealing handle, cut the package by sliding the blade holder from a side to another and taking it back to the initial position (Fig. 3).
- 5 Lift up the handle (Fig. 4) and take the package.
- 6 After using, press the On/Off button until the leds turn off.

Note: After 30 minutes with no use, the equipment is automatically turned off.

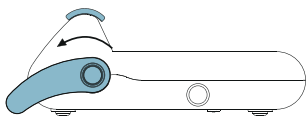
Fig. 2

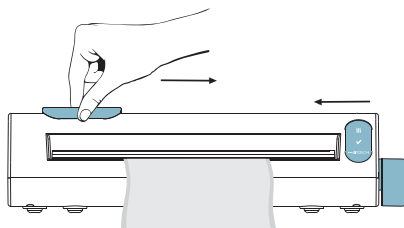
Fig. 3

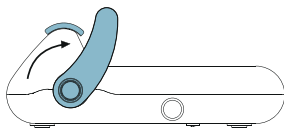
Fig. 4

6. Technical data

Model	Voltage / Hz	Power	Sealing Area (width)	Sealing Thickness	Sealing Temperature	Dimensions (cm)	Gross Weight Net Weight
TS-12	110V - 240V 50/60 Hz	180 W	30 cm	12 mm	175°C to 180°C	(package) 45 x 12,5 x 31 (sealer) 44,2 x 12,5 x 30,5	3,3 kg 2,9 kg

7. Preventive maintenance

In order to avoid the risk of injuries such as accidental burns, unplug the equipment and wait for the sealing bar to cool down before performing any maintenance procedure.

Cleaning

For an efficient cleaning, which must be done on a daily basis, use on the external side of the equipment a damp cloth with biodegradable neutral detergent, after it, clean with a cloth dampened with alcohol 70% or peracetic acid at 1%. Periodically search for paper stuck in the blade's guide slot.

Replacement of blades

BE CAREFUL WHEN HANDLING THE BLADES! Place the blade holder at the center position, push the guide (Fig. 5) and take the blade holder out (Fig. 6). Support the holder on the guide (Fig. 7) and take out the screws. Take out the old blades (Fig. 8), place the new ones (n° 23) and repeat the process inversely. To put back the blade holder, place it at the center position, verify if the tip of the blades is centered to the guide slot and press it down.

Fig. 5

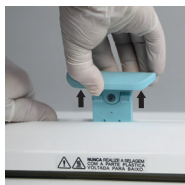
Fig. 6

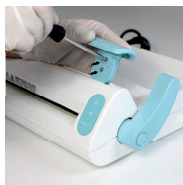
Fig. 7

Fig. 8

 **Never perform sealing with the plastic side downwards to avoid trouble with access of plastic material on the heater.**

8. Warranty terms

The manufacturer warrants the Sealer for 12 months against any manufacturing defect from the date of the purchase receipt.

The warranty will be voided immediately in cases of:

- Problems arising from natural causes (such as floods, lightning, landslide, etc.).
- Sinister (theft or robbery).
- Damage caused by accidents, such as: drops, power outage, etc.
- Use of materials that are not in accordance to the type of equipment, as described in the instruction manual.
- Removal and/or tempering of the serial number shown in the identification label of the product.
- Signs of tampering and/or blotted out data on the purchase or service receipt of the equipment.
- Signs of external violation of the product or broken factory seal.
- Use of accessories and materials not specified by the manufacturer.
- Modifications of the equipment performed by the customers themselves or at their own request.
- Repairs performed by technicians who are not part of the manufacturer Authorized Service Network.
- Noncompliance with any measure or caution recommended in the instruction manual of the product.
- Lack of grounding or grounding not in accordance with the current standards of the country.

The Heating element is not covered by warranty if there's plastic material on the heater. Cutting blades are not covered by warranty.

9. How to proceed in case of need for repair

If you have any problems, contact your local retailer or an authorised representative to schedule an evaluation or repair of your appliance.

You will need to be able to provide the model of your appliance, voltage, serial number and the date of manufacture on the identification label on the underside of the device, as well as a description of the problem.

10. Certification

Thermosealer TS-12 complies with strict international directives and standards ISO13485 and CE (Directive 2014/30/UE – European Standard EN55014-1 and EN55014-2).

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