


Designations	CMA ACCESSORIES									
References	<table border="1"> <tr> <td>3140987</td> <td>SEQUENCEUR CMA</td> </tr> <tr> <td>3140985</td> <td>ENDO-CLEAN SUPPORT OVAL NOIR CMA</td> </tr> <tr> <td>3140984</td> <td>ENDO-CLEAN SUPPORT OVAL BLEU CMA</td> </tr> <tr> <td>3140986</td> <td>DISQUES ENDO-CLEAN DIA. 6,5 CM CMA (25)</td> </tr> </table>	3140987	SEQUENCEUR CMA	3140985	ENDO-CLEAN SUPPORT OVAL NOIR CMA	3140984	ENDO-CLEAN SUPPORT OVAL BLEU CMA	3140986	DISQUES ENDO-CLEAN DIA. 6,5 CM CMA (25)	
3140987	SEQUENCEUR CMA									
3140985	ENDO-CLEAN SUPPORT OVAL NOIR CMA									
3140984	ENDO-CLEAN SUPPORT OVAL BLEU CMA									
3140986	DISQUES ENDO-CLEAN DIA. 6,5 CM CMA (25)									
Visuals										
Description	<p><u>SEQUENCEUR CMA:</u> Endodontic sequencer with 8 perforations and 2 graduated scales, providing sterilisation, storage and support for a sequence of endodontic Instruments.</p> <ul style="list-style-type: none"> • Closed, the CMA sequencer serves as a support for the sterilization and storage of endodontic Instruments. • Open, the sequencer serves as a support for easy handling of the instruments. <p><u>ENDO-CLEAN SUPPORT OVAL CMA:</u> Plastic holder for cleaning and storing endodontic instruments during treatment.</p> <ul style="list-style-type: none"> • For use with foam discs and compresses. • It can be filled with disinfectant solution. <p><u>DISCS ENDO-CLEAN CMA</u> Box of 25 foam discs, diameter 6,5 cm. Single-use foam discs for use with disinfectant solution.</p>									
Characteristics	<ul style="list-style-type: none"> • The sequencers and plastic holders are reusable after sterilization. • The sponges are single use. 									
Disinfection and sterilization	<p><u>Sequencer & Endo-clean support oval:</u> Before use, sterilise in an autoclave at 134°C for 18 minutes. The product may change colour slightly, but this will not affect its use or performance.</p> <p><u>Discs endo-clean:</u> For single use only.</p>									
Storage	No specific requirements.									
Legal information	Class I medical device under MDR 2017/745. CE Marking. Manufacturer LARIDENT S.R.L.									