



DENTO-VIRACTIS SURFACES 77

DENTO-VIRACTIS

Description	<p>Dento-Viractis 77 is recommended for cleaning and disinfection by wet wiping.</p> 
Scope of application	<p>For use on non-disposable medical devices, which are not sterilizable and non-immersible.</p>
Characteristics	<ul style="list-style-type: none"> ✓ Disinfectant Cleaner with a broad spectrum of activity ✓ Easy to use by wet wiping ✓ Pleasant odour
Commercial presentation	<p>1 liter bottle with sprayer The 5 liter can The box of 120 wipes The refill of 100 wipes</p>

Antimicrobial activity			Contact time
Bactericidal	EN 1276	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Escherichia coli	5 min
	EN 13697	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Escherichia coli	5 min
	EN 13727	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	1 min
	EN 14561	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Acinetobacter baumannii	5 min
	EN 16615*	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	5 min
Fongicidal	EN 1650	Candida albicans, Aspergillus brasiliensis (niger)	5 min
	EN 13697	Candida albicans, Aspergillus brasiliensis (niger)	5 min
	EN 16615*	Candida albicans	5 min
	EN 13624	Candida albicans / Aspergillus brasiliensis (niger)	1 min / 5 min



DENTO-VIRACTIS SURFACES 77

DENTO-VIRACTIS

	EN 14562	Candida albicans, Aspergillus brasiliensis (niger)	5 min
Virucidal	EN 14476	Norovirus	30sec
	According to EN 14476	Adenovirus, Vaccine ***	5 min
		Poliovirus	3 min
		Rotavirus	5 min
		HIV	30 sec
EN 16777	Vaccine***, Adenovirus / Norovirus	5 min	

Sporicidal	NF T 72-281**	Bacillus subtilis	5 min
	EN 17126	Clostridium difficile	15 min

* Specific protocol for use in wipes.

** Specific standard for use in sprayer (followed or not by a mechanical action)

*** The Vaccine Virus is the representative virus of the enveloped viruses presented in the annexes of the EN 14476+A2 (2019) and EN 17111 standards, such as Hepatitis B virus (HBV); Hepatitis C virus (HCV); Human immunodeficiency virus (HIV); Herpesviridae (Herpes virus); Coronavirus; Influenza, rabies, rubella, measles measles; ...

Composition	Wipe impregnated with a liquid solution. PT2/PT4, ready to use.
Instructions for use	<p>Instructions for a spray :</p> <ol style="list-style-type: none"> 1. Spray in order to adequately cover the surface (14mg/cm²) according to NF T 72-281) or on a clean non-woven cloth. 2. Clean thoroughly the surface to be treated. 3. Leave the product to act 5 minutes (15 min for C. difficile). Do not rinse (except food contact surfaces). 4. Use a fresh non-woven cloth for each equipment. <p>Instructions for wipes :</p> <p>Clean thoroughly the surface to be treated.</p> <ol style="list-style-type: none"> 1. Leave the product to act 5 minutes (15 min for C. difficile). Do not rinse (except for food contact surfaces). 2. Use a fresh wipe for each equipment. 3. Reseal properly the box after use.



DENTO-VIRACTIS SURFACES 77

Security phrases:

DANGER Contains D-LIMONENE. May produce an allergic reaction. Harmful to aquatic life with long lasting effects. Highly flammable liquid and vapour. Causes serious eye irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Wear protective equipment of eyes. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Store in a well-ventilated place. Keep cool. Avoid release to the environment. Dispose of contents/container as hazardous waste. Do not reuse container. CE Marking obtained in 2019.

Labelling:



CE 0459 Marking obtained in 2019. Class IIb DM. Version of the sheet: 2.0 09/2021

Distribué par :

CADENCE

2 bis, chemin du loup
93290 Tremblay-en-France
FRANCE



SODEL - ISO 9001/13485

190 rue René Barthélemy, Pôle d'activités
de l'Espérance, 14100 Lisieux, FRANCE.
Tel: +33 (0)2 31 31 10 50
www.sodel-sa.eu - www.alkapharm.fr

DENTO-VIRACTIS