




DENTO-VIRACTIS 75

MEDICAL DEVICES

DENTO-VIRACTIS

Ready to use !

Description	Dento-Viractis 75 without alcohol is recommended for cleaning and disinfection by wet wiping .	
Scope of application	For use on non-disposable medical devices, which are not sterilizable and non-immersible.	
Characteristics	<ul style="list-style-type: none"> ✓ Disinfectant Cleaner with a broad spectrum of activity ✓ Easy to use by wet wiping ✓ Without flavor ✓ PH : neutral (7,00+/-0.25) 	
Commercial presentation	750 ml spray 5 L canister	

Antimicrobial activity, in dirty conditions, 20°C			Contact time
Bactericidal	EN 13727	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	5 min
	EN 14561	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	5 min
	EN 1276	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Escherichia coli	5 min
	EN 13697	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Escherichia coli	5 min
	NF T 72-281**	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Escherichia coli	5 min
Yeasticidal	EN 13624	Candida albicans	5 min
	EN 14562	Candida albicans	5 min
	EN 1650	Candida albicans	5 min
	EN 13697	Candida albicans	5 min
	NF T 72-281**	Candida albicans, Aspergillus fumigatus	5 min
Virucidal	EN 16777	Vaccine ***	5 min
	EN 14476	Adenovirus, Norovirus, Hépatite B, Hépatite C, HIV, Rotavirus	5 min
Sporicidal	NF T 72-281**	Bacillus subtilis	5 min
	NF T 72-281**	Clostridium difficile	5 min



DENTO-VIRACTIS MEDICAL DEVICES 75

DENTO-VIRACTIS

Composition	Liquid preparation, PT2/PT4, ready to use. DIDECYL DIMETHYLAMMONIUM CHLORIDE n°CAS 7173-51-5 at 0.25 g/kg. ALKYL (C12-16) DIMETHYLBENZYLAMMONIUM CHLORIDE n°CAS 68424-85-1 at 0.95 g/kg.
Direction for use	Active on contact with the medical device. Use biocidal products with care. Read the label and product information before use. <ol style="list-style-type: none">1. Spray in order to adequately cover the surface (0,43g/cm² according to NF T 72-281) or on a clean non-woven cloth.2. Clean thoroughly the surface.3. Leave the product to act 5 minutes. Do not rinse (except for food contact surfaces).4. Use a fresh non-woven cloth for each equipment.

** Standards realized on sprayers (followed or not by a mechanical action). *** : The Vaccine Virus is the representative virus of the enveloped viruses presented in the annexes of the EN standards. 14476+A2 (2019), EN 16777 and EN 17111, e.g.: Hepatitis B virus (HBV); Hepatitis C virus (HCV); Human Immunodeficiency Virus (HIV); Herpesviridae (Herpes virus); Coronavirus; Influenza, rabies, tuberculosis and malaria viruses. rubella, measles; ...

CE0459 Marking obtained in 2019. Version 2.0 of 09/2021
Class IIb medical device
Professional use

Distribué par :
CADENCE
2 bis, chemin du loup
93290 Tremblay-en-France
FRANCE



SODEL - ISO 9001/13485
190 rue René Barthélemy, Pôle d'activités
de l'Espérance, 14100 Lisieux, FRANCE.
Tel: +33 (0)2 31 31 10 50
www.sodel-sa.eu - www.alkapharm.fr