	EC REP	CE
Files	Maillefer Instruments Holding Sàrl Chemin du Verger 3, CH-1338 Ballaigues, Switzerland	Dentsply DeTrey GmbH De-Trey-Straße 1, D-78467, Konstanz, Germany	2797
Gutta-Percha & Absorbent Points	Dentsply Tulsa Dental Specialties 608 Rolling Hills Drive, Johnson City, TN 37604, USA	Dentsply DeTrey GmbH De-Trey-Straße 1, D-78467, Konstanz, Germany	2797
Irrigation Needle	Produits Dentaires SA Rue des Bosquets 18, CH-1800, Vevey	PD Dental EU 74200 Thonon-les-Bains, France	1639
AH Plus® Bioceramic Sealer	Maruchi 2-208, Medical Industry Complex Blidg., 42-10, Taejanggungdangil, Wonju-si, Gangwon-do, KR-26311, South Korea	Biogeri Am Pfeilshof 12, DE-22303 Hamburg, Germany	0197

INL/ BEN PTUL TIP 000 / 00/2021 - created 06/2021

Deep Shape

by 

Shaping

Cleaning

Obturation

MAILLEFER
ProTaper Ultimate™

 Dentsply
Sirona


MAILEFER

ProTaper Ultimate™


SHAPING

Motor settings: 400 rpm / 4-5.2 Ncm

STANDARD

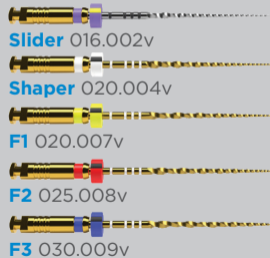

LARGER AND STRAIGHTER CANALS ONLY

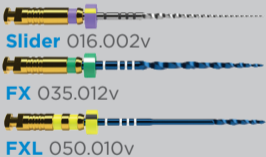

CLEANING

Upon removing any given file, clean and irrigate with the Irrigation Needle

Use SmartLite® Pro EndoActivator® to remove debris

Coming soon

OBTURATION

Absorbent Points
AH Plus® Bioceramic Sealer
Conform Fit® Gutta-Percha Points

ACCESSORIES

Dedicated manual version

SX
Orifice Opener