Scan for more information

Check out our learning platform Reliance U! www.RelianceU.tech

 **Reliance Orthodontic Products, Inc.**
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EMERGO EUROPE
Prinsessgracht 20, 2514 AP The Hague,
The Netherlands

**AUSTRALIAN SPONSOR
EMERGO AUSTRALIA**
Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

RELIANCE® METAL PRIMER

INSTRUCTIONS FOR USE

 **2797** **V2 MAR.2022**

 **RELIANCE®**
**ORTHODONTIC
PRODUCTS**

RelianceOrthodontics.com

EN - ENGLISH

INDICATIONS FOR USE: Reliance Metal Primer is intended for use as a conditioner of metal surfaces for bonding with an adhesive.

INSTRUCTIONS FOR USE:

1. Prophly and sandblast the metal surface. Rinse and dry thoroughly.
2. Apply 1 coat of Reliance Metal Primer on the metal surface area to be bonded. Wait 30 seconds.
3. Paint a thin coat of sealant on the conditioned metal surface. Apply the paste to the bracket base and position on the tooth. Light cure

Special Safety Notice: This device Brush applicators are single use devices. It is required that a new brush applicator is used for each application.

⚠ Flammable

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

🌡️ **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C).

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontics Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from use of or suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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