**Intended Use**

The Steam Incubator, manufactured for 3M ESPE, is designed and has been tested to be used with Attest™ Biological Indicators (BI) 1261P or 1262P, manufactured for 3M ESPE, and allows rapid, reliable monitoring of steam sterilization process. As a quick reference, the steam monitoring system is color-coded blue and brown, respectively. Use with other biological indicators may lead to incorrect results.

Each Attest incubator is pre-set at the appropriate temperature to promote growth of the test organism. The incubator's internal temperatures are rigidly controlled to 56°C +/-2°C (133°F +/-3°F) and automatically maintained. If a temperature measurement device is used to monitor the incubator, the sensor should be in contact with the metal heating block wall.

**Safety Information**

**⚠ WARNING**

To reduce the risk of electric shock:

- do not pour any liquids into the metal heating block holes
- do not immerse the incubator in any liquid
- use indoors only
- disconnect power cord before cleaning
- use only a disinfectant or mild detergent and wipe with a damp cloth
- there are no user serviceable parts
- the incubator must be returned to the manufacturer for repair
- make sure a properly rated power cord is installed on the incubator

To reduce the risk of using a load from a sterilization process failure:

- match the color on the exterior label of the incubator with the same color cap on the BI
- verify the temperature of the metal heating block
- use a control BI each day a test BI is incubated

There is a glass ampule inside the plastic vial of the biological indicator:

- crushing or excessive handling of the BI before cooling may cause the glass ampule to burst
- wear safety glasses and gloves when removing the BI from the sterilizer
- wear safety glasses when crushing the biological indicator
- handle the BI by the cap when crushing and tapping
- do not use your fingers to crush the glass ampule
- do not roll the biological indicator between fingers to wet the spore strip

**⚠ CAUTION**

To avoid possible environmental contamination, dispose of the incubator according to applicable governmental regulations

Avoid contact with the hot surface of the metal heating block

**Explanation of symbols**

-  Attention - Refer to accompanying documents
-  Caution - Risk of electric shock
-  Warning - Do not immerse
-  Warning - Do not pour liquids

**Environmental Operating Conditions**


Environmental Condition	Operating Condition	Units
Altitude	2500 (max)	meters
Operating Temperature	15 - 35	°C
Relative Humidity	35 - 80 (non-condensing)	%
Voltage Range	110/120 or 220-240	Volts AC
Frequency	50/60	Hertz
Current	0.25 or 0.13	Amps
Installation/Over Voltage	Category II	
Pollution Degree	2	

Attest Incubators	Attest Incubators	Sterilization Process Monitored	Attest Biological Indicators	Color Coding
<b>110/120 volt</b>	<b>220/240 volt</b>			
Model 116 (14 vials)	Model 118 (14 vials)	Steam	1262 & 1262P	Brown
Model 126 (28 vials)	Model 128 (28 vials)	(except 270°F/132°C gravity displacement)		
Model 130 top tier (14 vials)	Model 131 top tier (14 vials)			
133 +/- 3°F (56 +/- 2°C)	133 +/- 3°F (56 +/- 2°C)			
See above	See above	Steam gravity displacement 270°F/132°C	1261 & 1261P	Blue

**Instructions for use:**

1. After completion of the cycle and while wearing safety glasses and gloves, fully open the sterilizer door for a minimum of 5 minutes prior to removing the Attest biological indicator.
2. When the biological indicator is not contained in a test pack or other heat absorbing packaging material, remove the biological indicator from the sterilizer and allow to cool for an additional 10 minutes prior to crushing.
3. When the biological indicator is contained in a test pack or other heat absorbing packaging material, the test pack or any other heat absorbing packaging material should be removed from the sterilizer and opened up for 5 minutes to dissipate heat prior to removing the biological indicator. Then allow the indicator to cool outside the test pack for an additional 10 minutes prior to crushing.
4. Position the biological indicator in the incubator's metal heating block (See Figure 1). Place the bottom of the indicator into the metal heating block so that the indicator is at an angle of approximately 45° and the cap is above the metal block.
5. Push the indicator straight back (See Figure 2). This crushes the ampule and activates the indicator.
6. Push the activated indicator down to seat it in the metal heating block (See Figure 3). Be sure that the cap remains above the metal block.

Refer to the Attest biological indicator package insert for further instructions for use, including the interpretation of results and use of controls.

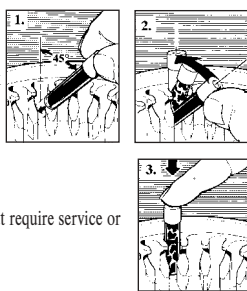**Repair and Replacement**

Repairs must be done by a qualified service person. Attest incubators that require service or repair should be returned to the following address (USA only):

3M Health Care Service Center  
 Suite 200, Building 502  
 3350 Granada Avenue North  
 Oakdale, MN 55128

Outside the United States, contact the local subsidiary. In Canada, contact 3M Canada, P.O. Box 5757, London, Ontario, N6A 4T1. 1-800-563-2921

U.S.A. Service Center Help-line: 1-800-292-6298  
 FAX: 1-800-770-8016

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Warranty**

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

**Limitation of Liability**

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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