Technical Specification Datasheet – Customer Certification of the finished Device

Manufacturer: PAJUNK® GmbH Medizintechnologie Karl-Hall-Strasse 1 78187 Geisingen Germany		Distributor: <i>[Authorized Stamp here]</i>
Manufactured according to ISO13485, ISO14971, GMP		
The product and manufacturing process is free of Latex.		
Project	R15/4010	
Device	Syringe UNIJECT K Dental Syringe; Hooklets, swing-out, crutch/ thumb ring, chromium plated, without needle, reusable	
Regulatory Information	RL93/42 & RL2007/47/EEC (MDD) class I rule 5-1 GMDN: 35969 Syringe, dental, anaesthesia	
EC Certificate	51268-17-01	
Notified Body	DEKRA Certification GmbH, Stuttgart, NBC: 0124	
<input type="checkbox"/> US-510(k)	510(k) exempt	
Listing.No.	Not available	
Regulatory Inf.	Not available	
Device Description	Different design variants with stick-on fixture and hinged retainer technology are available. Syringes with stick-on fixture have the advantage, a fast exchange during the treatment with the cartridge holders, which was filled in advance, is possible. Both techniques support the manual aspiration by means of three extensible aspiration hooks as well as self-aspiration. They are suitable for the use of conventional 1,8ml cylinder cartridges and for cannulas with a metric tread.	
Intended Use	To hold dental cartridges (anesthesia / ointment) in combination of special dental cannulas	
Patient population	Without limitation	

	Procedure	<ol style="list-style-type: none"> 1. The cannula is screwed direct on the syringe body. 2. The setting with the three aspiration hooks enables a stable three-point-anchorage in the hollow plug of the cartridge and safes a free of tilt guidance during injection and aspiration. 3. By turning the knurled screw to the right the hooks are moved out, by turning to the left moved in. 4. Retire the piston rod. 5. Swing out the cylinder cartridge holder perpendicular. 6. Insert the cartridge. 7. Swing out back the holder in to the longitudinal axis 8. Therefore retire the piston rod complete, against the feeder resistance. 9. Release the feeder resistance. The cartridge slips, centered by a guide ring, will be positioned in the right function position. 	
	Picture (not for scale)		
	Compatibility	<p>The dental cartridge syringes are compatible with the M6x0,75 mm or BSW 0,214-40TPI thread at the tip with all dental cannulas according to standard DIN EN ISO 9997 or DIN EN ISO 7885. In addition, all cartridges, which meet the specification of DIN EN ISO 11499, can be used in the dental cartridge syringes.</p>	
	Materials	Chromed brass	
	Packaging	Single device:	Card board box with foam inlay
		Multiple Units:	Card board box
	Additional information		Does not contain latex
			Does not contain Phthalates

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		Storage and Shipment: Humidity Range 20-65%
	WARNING! For all obese patients select appropriate and corresponding dimensions (diameter and length) of the instrument.	
Released/Update	September 07 th 2015 	
	Christian Quass, Director Regulatory Affairs, September 07 th , 2015	

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Indicates Lot - by consecutive numbering per sterile batch - by code "AB" for "Month.Year" per non sterile device