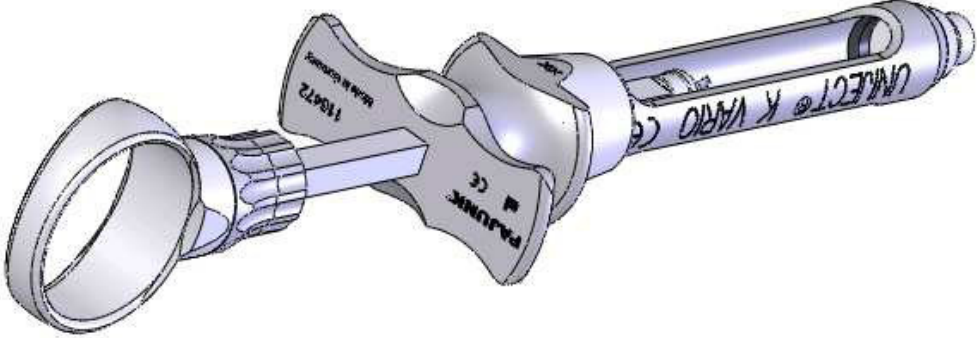
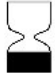
## Technical Specification Datasheet – Customer Certification of the finished Device

<b>Manufacturer:</b> <b>PAJUNK® GmbH Medizintechnologie</b> Karl-Hall-Strasse 1 78187 Geisingen Germany		<b>Distributor:</b> <i>[Authorized Stamp here]</i>
Manufactured according to ISO13485, ISO14971, GMP		
The product and manufacturing process is free of Latex.		
<b>Project</b>	R15/4012	
<b>Device</b>	<b>Syringe UNIJECT K VARIO</b> Dental Syringe; Hooklets, slip-on, crutch/ thumb ring, chromium plated, without needle, reusable	
<b>Regulatory Information</b>	RL93/42 & RL2007/47/EEC (MDD) class I rule 5-1 GMDN: 35969 Syringe, dental, anaesthesia	
<b>EC Certificate</b>	51268-17-01	
<b>Notified Body</b>	DEKRA Certification GmbH, Stuttgart, NBC: 0124	
<input type="checkbox"/> <b>US-510(k)</b>	510(k) exempt	
<b>Listing.No.</b>	Not available	
<b>Regulatory Inf.</b>	Not available	
<b>Device Description</b>	Different design variants with stick-on fixture and hinged retainer technology are available. Syringes with stick-on fixture have the advantage, a fast exchange during the treatment with the cartridge holders, which was filled in advance, is possible. Both techniques support the manual aspiration by means of three extensible aspiration hooks as well as self-aspiration. They are suitable for the use of conventional 1,8ml cylinder cartridges and for cannulas with a metric tread.	
<b>Intended Use</b>	To hold dental cartridges (anesthesia / ointment) in combination of special dental cannulas	
<b>Patient population</b>	Without limitation	

	Procedure	<ol style="list-style-type: none"> <li>1. The cannula is screwed direct on the syringe body.</li> <li>2. The setting with the three aspiration hooks enables a stable three-point-anchorage in the hollow plug of the cartridge and safes a free of tilt guidance during injection and aspiration.</li> <li>3. By turning the knurled screw to the right the hooks are moved out, by turning to the left moved in.</li> <li>4. Retire the piston rod.</li> <li>5. Slip off the cartridge holder by moving lateral out of the slip-on mechanism of the syringe.</li> <li>6. Insert the cartridge.</li> <li>7. Slip-on back the holder in to the slip-on mechanism.</li> <li>8. Thereby retire the piston rod complete, against the feeder resistance.</li> <li>9. Release the feeder resistance. The cartridge slips, centered by a guide ring, will be positioned in the right function position.</li> </ol>	
	Picture (not for scale)		
	Compatibility	<p>The dental cartridge syringes are compatible with the M6x0,75 mm or BSW 0,214-40TPI thread at the tip with all dental cannulas according to standard DIN EN ISO 9997 or DIN EN ISO 7885. In addition, all cartridges, which meet the specification of DIN EN ISO 11499, can be used in the dental cartridge syringes.</p>	
	Materials	Chromed brass	
	Packaging	<b>Single device:</b>	Card board box with foam inlay
		<b>Multiple Units:</b>	Card board box
	Additional information		Does not contain latex

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		Storage and Shipment: Temperature Range 20 ±10 °C
		Storage and Shipment: Humidity Range 20-65%
	<b>WARNING!</b> <b>For all obese patients select appropriate and corresponding dimensions (diameter and length) of the instrument.</b>	
Released/Update	September 07 <sup>th</sup> 2015  	
	Christian Quass, Director Regulatory Affairs, September 07 <sup>th</sup> , 2015	