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- UK** Dental injection needles, with patented scalpel bevel, sterile, single-use, stainless steel, siliconized, dental needle with bevel marking

INDICATION

- The needle are indicated for local dental anaesthetic injections.

CONTRAINDICATION

- Hypersensitivity to any component.

INSTRUCTIONS FOR USE

- Break the sterility guarantee seal just before use by turning the two plastic caps in opposite directions.
- Remove the rear cap.
- Screw the pre-threaded needle onto the pre-threaded hub of the syringe.
- Remove the plastic cap just before the injection is administered.
- After use, unscrew using a safe removing procedure complying with local practices/regulations.

DISPOSAL

- Discard the used needle into a puncture-resistant container in accordance with applicable local medical waste regulations.

WARNINGS

- In the event that the sterility guarantee seal is already broken, do not use the needle.
- Never recap needles by hand. Recapping needles by hand or recapping bent or broken needles can cause unintended needle sticks.
- Do not bend or apply otherwise stress needles as serious injuries to you and/or your patient can occur.
- Do not insert needle to hub during injections, as needles can break and become lodged in patient’s tissue, potentially causing serious permanent injury.
- Avoid excessive pressure and excessive needle movement during injection as this may cause needle breakage , which may result in serious injury to you and/or your patient.
- The needle is a single-use device, do not reuse the needle on a different patient to avoid cross contamination.
- Do not reuse the device with several cartridges to avoid performance issues (cannula penetration) and cannula breakage.
- Do not re-sterilise otherwise sterility and performance of this device cannot be guaranteed.
- Be especially vigilant when using a needle in a patient who appears to be apprehensive.

CAUTIONS

- Store in original packaging until ready to use.

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