FR

COMPOSITION

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EN

COMPOSITION

Zinc oxide, Zinc Sulphate, Excipient

INDICATIONS

Septo-pack is a malleable self-hardening paste, with fibres in its mass.

En periodontology: Septo-pack is a periodontal dressing for the protection of the wounds after deep curettage or periodontal surgery. It can also be used as a neutral medium to drugs to keep them in place at the gingival, tooth or alveolar level.

In operative dentistry: Septo-pack is also a temporary filling material for inlay or onlay cavities or a gingival retraction material for cervical cavities.

CONTRAINDICATIONS

Septo-pack is contraindicated on patients with a known allergy to one of the components of the product.

INSTRUCTIONS FOR USE

Take a small quantity of product from the jar with a dry, sterile spatula and do not wet the material throughout manipulation. Tightly close the jar after use.

In periodontology: After forming the material into a small roll of the required length and thickness, apply Septo-pack onto the area to be covered. Press the material into the interproximal spaces to anchor it and to limit the gingival compression. Ensure that the dressing does not extend on the occlusal surfaces and beyond the immediate area to be protected. The material should not extend beyond the muco-gingival margin to avoid mechanical irritation of the alveolar mucoa. Working time in the mouth is only 2 or 3 minutes following application. Hardening is complete in half an hour. If a prolonged protection of the wound area is needed, it is better to renew the material after 7 days.

In operative dentistry: A thin layer of non-greasy lubricant should be applied onto the cavity walls to facilitate the subsequent removal of the temporary restoration. Delicately dry the cavity with air. Apply the required quantity of Septo-pack in the cavity with a spatula.

To remove this temporary filling in one piece, use a scaler or any other instrument with a sharp point. Eliminate any material left before permanently sealing the inlay or onlay.

PRECAUTIONS FOR USE

Xi - Irritating to eyes.

N - Dangerous for the environment

Very toxic to aquatic organisms, may cause long-term adverse effects in the aquatic environment. Avoid release to the environment. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

This material and its container must be disposed of as hazardous waste.

PRESENTATION

Jar containing 60 g of paste.

EXPIRY DATE

Do not use after expiry date.

BATCH NUMBER

Batch number is on the packaging.

For dental professional dental use only

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