SS_ONE

SS_KISS

MANUALE D'INSTALLAZIONE_ITA
INSTALLATION GUIDE_ENG
MANUAL DE INSTALACIÓN_ESP

Capitoli – Chapter – Capítulos

SS_ONE/SS_KISS

MANUALE INSTALLAZIONE	p.1
INSTALLATION GUIDE – ENG	p.20
MANUAL DE INSTALACIÓN – ESP	p.39
SCHEMI TECNICI _ TECHNICAL DIAGRAMS DIAGRAMAS TÉCNICOS	p.58
CERTIFICATI _ CERTIFICATES _ CERTIFICADOS	p.62
GARANZIA _ WARRANTY _ GARANTÍA	p.69
CERTIFICAZIONE DI CONTROLLO CONTROL CERTIFICATION _ CERTIFICACIÓN DE CONTROL	p.71
REGISTRO VERIFICHE ELETTRICHE REGISTER OF ELECTRICAL TESTS REGISTRO DE PRUEBAS ELÉCTRICAS	p.73
ASSISTENZA _ ASSISTANCE _ ASISTENCIA	p.75
SCHEDA DI INSTALLAZIONE _ INSTALLATION SHEET FICHA DE INSTALACIÓN	p.77

ITALIANO

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Indice

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INSTALLATION GUIDE

Index

1 _ Precautions for Installation	22
1.1 _ Environmental Conditions	23
1.2 _ Disposal at End-of-Life	23
1.3 _ Safety Warnings	23
1.4 _ Electrical, Hydraulic and Pneumatic Considerations	25
2 _ Tools and Instruments	27
3 _ Conditions for Installation	28
4 _ Specifications on packaging	32
5 _ Installation	33
5.1 _ Levelling	33
5.2 _ Installing of the Backrest	34
5.3 _ Installing of the Operating Lamp	35
5.4 _ Installing of the Basin	36
6 _ Tests and Adjustments	37
7 _ Serial Numbers	37
8 _ Classifications and Reference Regulation	38
9 _ Label of the Product	38

1 _ Precautions for Installation

Before installing the unit, it is necessary to:

- Make sure that the floor is clean and safe to support the unit.
- Check that all the connections are ready.
- Make sure to let the incoming water and air come out before connecting the unit, so as to prevent any possible stagnant dirt inside the tubes to enter the machine.

Before installing the device, read this guide carefully:

- The input voltage is V AC220-230, 50/ 60 Hz , or AC110-120 V, 50/.
- Make sure that the unit is specifically electrically grounded.
- Make sure that the dental units is well fixed to the ground so as to prevent a possible overturning.
- Only fuses, accessories and spare parts admitted and authorized by Simple & Smart can be used on this device.
- Do not use this equipment in environments with high temperature, high humidity or placed on sloping floor.
- This unit must not come into contact with any object while the chair is moving.
- This device should not be used in environments where are present strong magnetic or electromagnetic fields.
- In case of disputes, it is considered as reference manual, the italian version or the English one. Any other translation, made by third persons, will not be taken into consideration.

1.1 _ Environmental Conditions

The equipment must be installed in environments that comply with the following conditions:

- temperature between 10 and 30° C;
- relative humidity between 45 and 75%;
- atmospheric pressure from 700 to 1060 hPa;
- pressure of the air entering the equipment between 0,5-0,8 hpa;
- hardness of the water entering the equipment not exceeding 60 mg/L;
- pressure of the water entering the equipment between 0,2-0,4 hpa;
- temperature of the water entering the equipment not exceeding 25° C.

1.2 _ Disposal at end-of-life

Pursuant to Directives 2002/95/ EC, 2002/96/ EC e 2003/108/ EC, regarding the reduction of the use of hazardous substances in electrical and electronic equipments, as well as waste disposal, it is required not to dispose these equipments as municipal waste, carrying out a separate collection.

Illegal disposal of this equipment entails the application of the penalties defined by each National Laws.

1.3 _ Safety Warnings

The weight of the dental unit, including a patient weighing 135 kg, is equal to approximately 340 kg/mq. A technician, non authorized by Simple & Smart, who proceeds to modify the product by replacing parts or components with other than those used by the manufacturer assumes a responsibility equivalent to that of the manufacturer.

SIMPLE&SMART srl accepts no responsibility for damage to property and persons if not respected this clause.

The Maximum permissible load on the chair is 135 kg. This weight should **NEVER** be exceeded.

Connections to external instruments.

The device can be electrically connected only to other instruments with the CE marking.

Electromagnetic interference.

The use, in the studio or in close proximity to electric equipments not compliant to legislation I.E.C. 60601-1-2, may cause electromagnetic or other interference, leading to a malfunction of the dental unit. In these cases it is recommended to turn off the power of the dental unit before using such equipment.

Replacement of the burrs

Operate the release devices of the turbines and the contrangles only when the burr is completely stopped. Otherwise the locking system deteriorates and the burrs may be accidentally released causing injuries. Use only high quality burrs, that have a connection stem with a calibrated diameter. To control the status of the locking device, check every day, at the beginning of the work, that the burr is firmly locked to the instrument. Defects in the locking system due to incorrect use are easily detectable and are not covered by warranty.

Patients with cardiac pacemaker and/or hearing aids.

In the treatment of patients with cardiac pacemaker and/or hearing aids, have to be considered the possible effects of the tools used on the cardiac pacemaker and/or hearing aid. For this purpose, please refer to the technical-scientific literature on the subject.

Implantology

In case in which the unit is employed for dental implantology, using autonomous equipments aimed to this intervention, it is recommended to turn off the power of the unit, in order to avoid possible unwanted movements caused by faults and/or accidental activation of the movement controls.

Before leaving the clinic, turn off the water supply of the studio and the general switch of the equipment.

The device is not protected against ingress of liquids (IPX 0).

The device is not suitable to be used in the presence of a mixture of flammable anesthetic gases with oxygen or nitrous oxide.

The equipment must be guarded and preserved in perfect efficiency. The manufacturer disclaims any liability (civil and criminal) for any abuse, careless use or improper use of the device.

The equipment must be used only by authorized personnel (medical and paramedical) adequately trained.

The equipment must always be supervised, when turned on or prepared to the activation, in particular, should never be left unattended in the presence of minors/incapable persons or, in general, unauthorized personal.

Any accompanying personnel must remain outside of the area where the treatment is carried out and in any case under the operator's responsibility.

1.4 _ Electrical, Hydraulic and Pneumatic Considerations

In case the water supplied is particularly hard, it is useful foreseen a preventive treatment of the same. Be sure to have fulfilled the regulation EN 17171 against pollution of the public water supply.

INSTALLATION GUIDE

The user of the dental unit is responsible for the quality of the water and must, if necessary, adopt alternative measures, aimed to the maintenance of the same, in case the dental unit is used without disinfection system.

Make sure that the power supply of the equipment is provided with an external differential circuit breaker, with a capacity of 16° and 10mA threshold; that the electricity grid that supplies power to the unit has been adequately and efficiently grounded and that the installation of the system complies with the applicable regulations.

Be sure that the hydric and pneumatic supplies are equipped with turn off valves external to the unit.

The new models are equipped with a unique locking system for the ducts of inlet water, air and electricity.

The compressed air for the pneumatic supply have to be dehumidified and without oil suspensions.

Check the inlet air pressure to the turbines on the manometer located on the right side of the main unit. The indicator must **NEVER** enter the red band, as it may cause damages to the internal air pipes due to high pressure.


To control and monitor the inlet pressure, it is necessary to adjust the pressure regulating valve at the base of the chair (beside the water pressure regulating valve).

The necessary electrical tests of the unit must be performed by a qualified technician. The end customer is responsible to let a specialized technician perform a periodic electrical control on the unit.

2 _ Tools and Instruments

The installation requires the following tools: electric drill, scissors, pliers, wrench, Allen wrench, spirit level, hammer, Phillips and flat-tip screwdrivers.


Any additional tools are at the installer's discretion.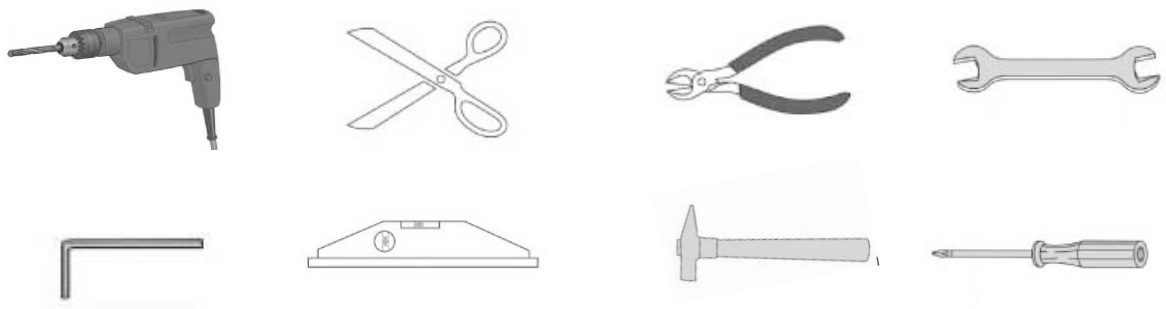

All the junctions necessary to the connection of the chair/unit, to the drains and the various connections are not provided by SIMPLE&SMART, but they must be properly prepared by the specialized technician, who has taken preventive vision of the installation site through an inspection.


3 _ Conditions for installation

Figure 1 _ Dental unit dimensions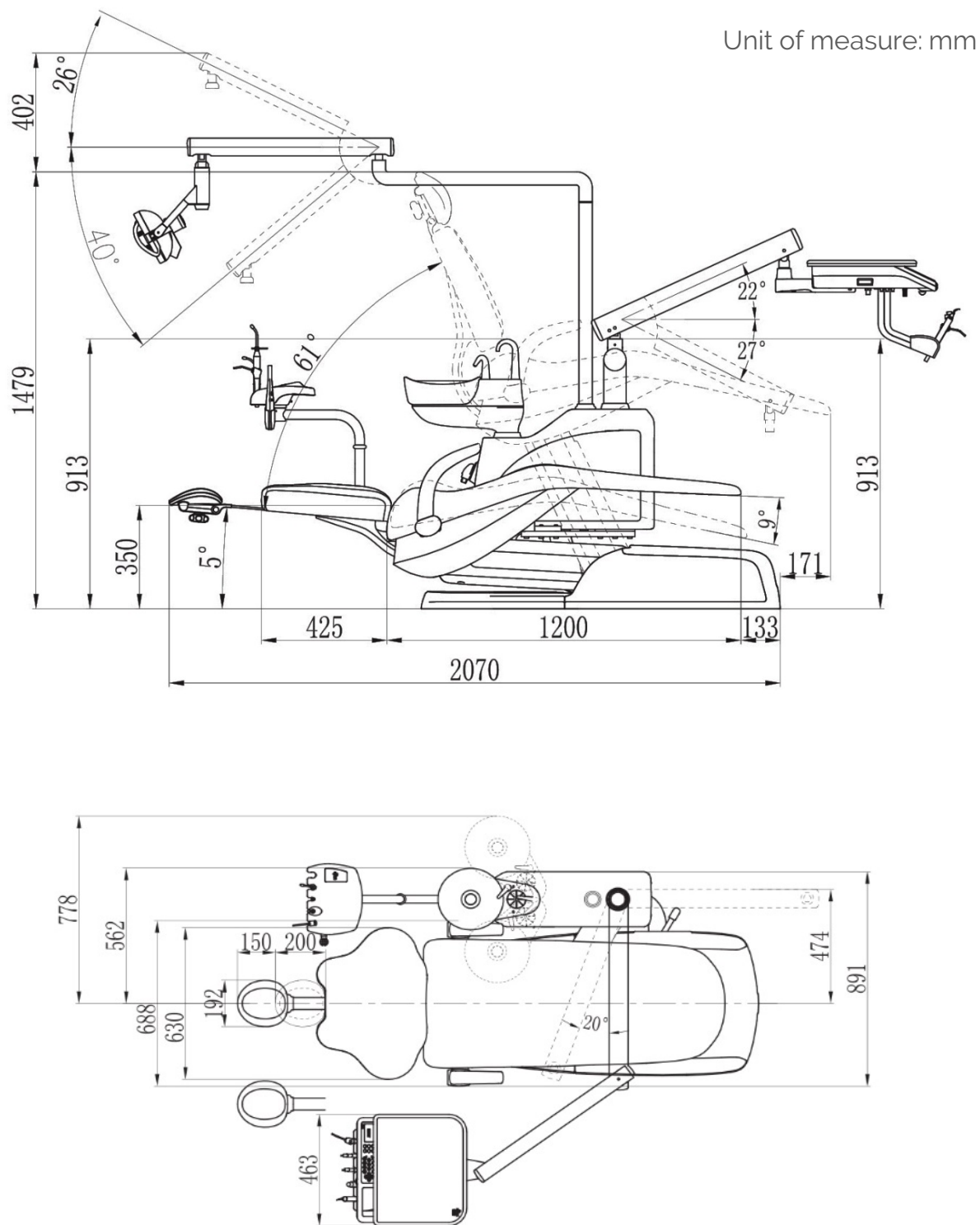


INSTALLATION GUIDE

Figure 2 – Position of the installation holes on the plate at the base of the chair. Chair position with respect to the front and side walls.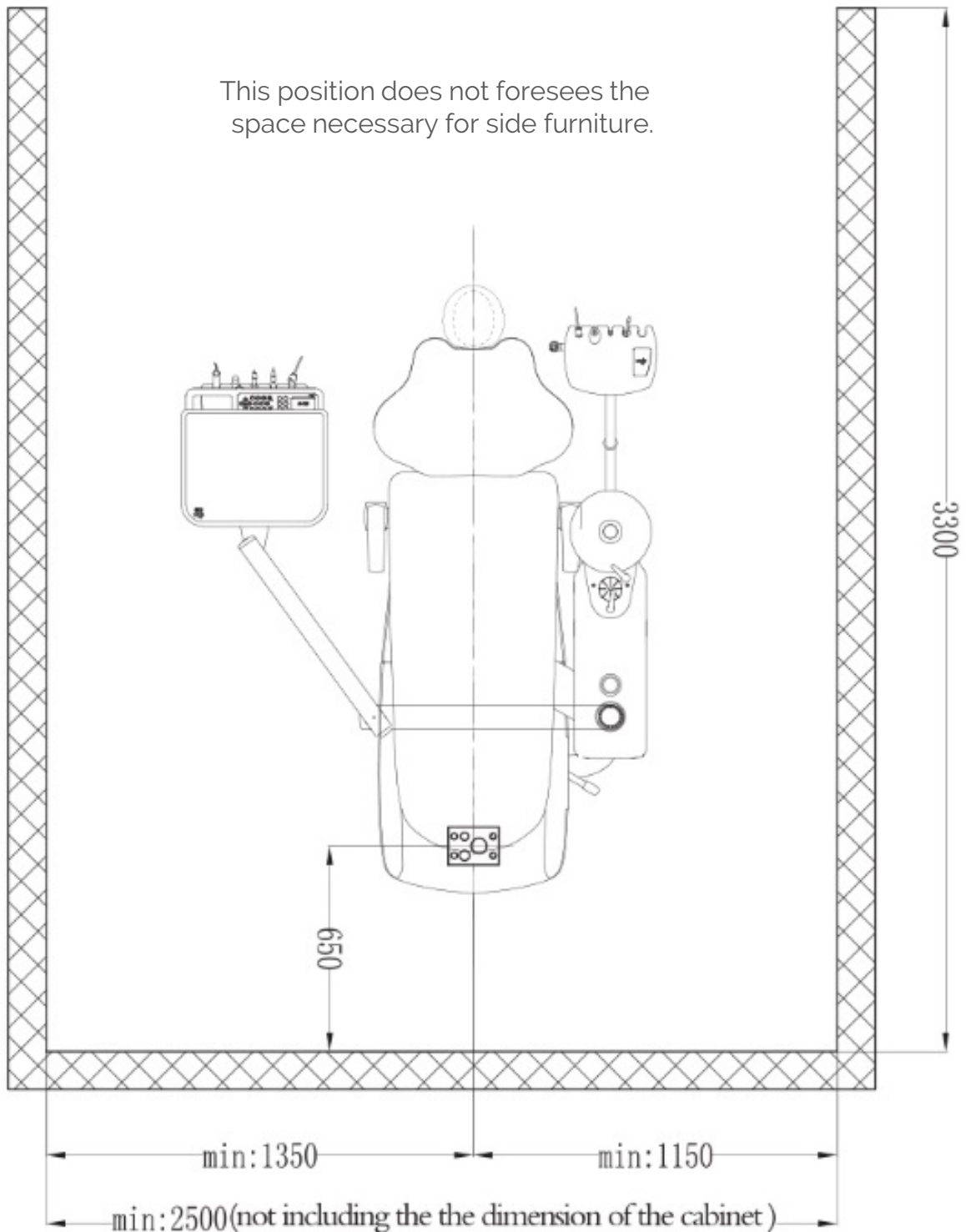

INSTALLATION GUIDE

Figure 3 - Recommendations for the preparation of pipes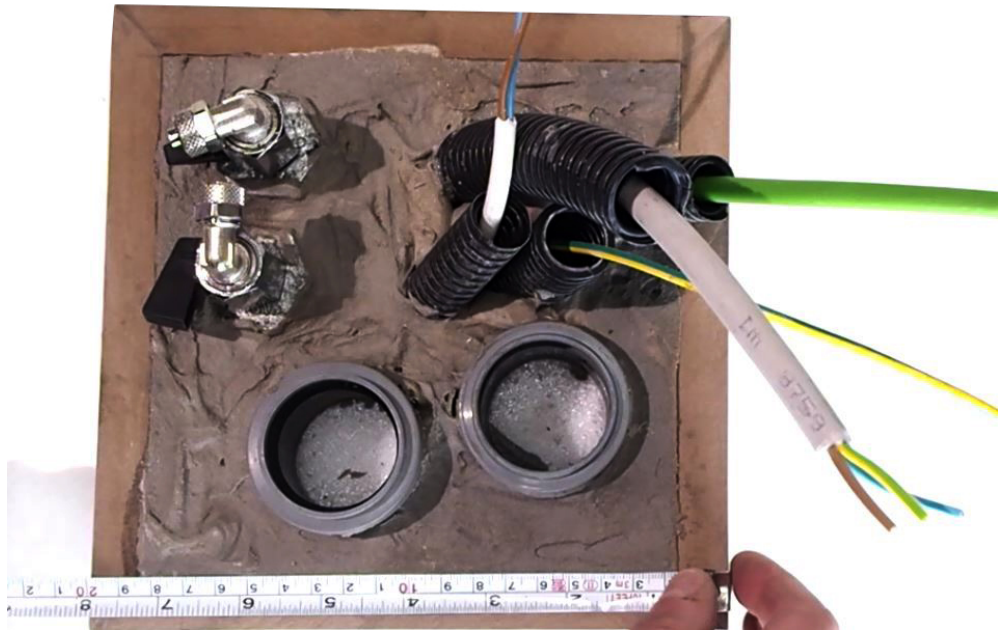

- Drainage Pipe: PVC Ø 40 mm. At the floor level.
- Suction Tube: Geberit Ø 40 mm in PVC. At the floor level. It is better to avoid 90° bends along the routing of pipes. This pipe must be connected to the suction machine.
- Water connection: tube Ø 6x8 mm multilevel. It is recommended that the floor connection tube is provided with a microball valve in the terminal part of the tube.
- Air connection: tube Ø 6x8 mm multistrato. It is recommended that the floor connection tube is provided with a microball valve in the terminal part of the tube.
- Power supply 220V (3 cables of 2,5mm)
- Grounding Cable
- Power supply for suction system (2 cables 2,5mm). There is no flow of electricity but it is used only as a on-off switch. It will be connected at one side to the unit and, at the other side with suction machine.

INSTALLATION GUIDE

All connections must be contained into a square space of 20 x 20 cm. **In frontal position below the dental unit.** The correct position is **under the patient's feet.**

The more the connections are close to each other and the easier it will be to cover them with the plastic structure of the unit.

Make sure that the connections are separated at least 40 cm from the front wall.


Before installing the unit in its final position, please take into account the length of the arm for the X-ray viewer. All the dimensions and guidelines provided above are only suitable for the installation of SIMPLE&SMART's unit. Please refer to the video posted on the website www.simplesmart.it for further information.

Input voltage: Voltage 220Volts.

Maximum Ampere 15A, Phase 1.

Frequency 50-60Hz.

Figure 4 – Pipes at floor level

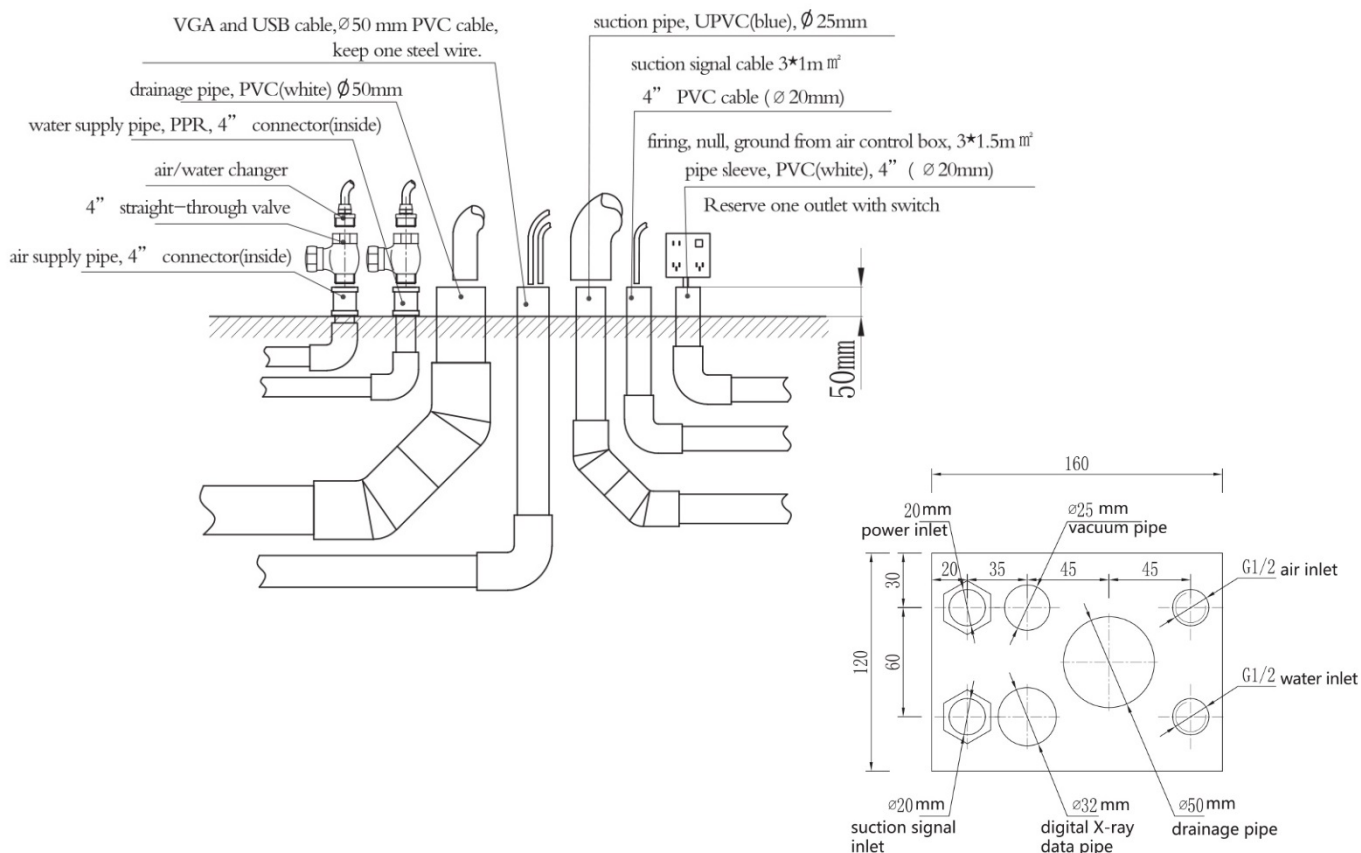4 _ Specifications on packaging

The unit is packed with scratch-resistant material and cartons. For the shipments we use specially built wooden cases. The packaging may be standard or optional, as shown in the figure below.

Figure 6 – Dimensions of the case for the shipment.

STANDARD
Wooden case
145 x 120 x 113
300 Kg

OPTIONAL
Wooden case
145 x 60 x 113 each
150 Kg each

5 _ Installation

For a complete and updated procedure, in order to proceed to a correct installation, we invite you to see the videos posted on SIMPLE&SMART's website:

www.simplesmart.it


After the equipment has been unpacked, make sure that all components and accessories are present and complete. In case that some components are damaged, the installation may result dangerous or the functioning incorrect.

- Plug the power cable into the socket.
- Turn on the power switch and lift the chair.
- Lift the front cover at the base of the unit and unscrew the grub screw of the rear cover, then remove the rear plastic cover.
- Remove the unit from the pallet and place it on the floor.
- Place the unit in the best position, install the whole unit by following these steps:

5.1 _ Levelling


Once you have verified that there are no slopes on the floor, place the unit and adjust the screws with the Allen Key, so as to have a perfect stability of the equipment.

Figure 7 - Levelling of the base plate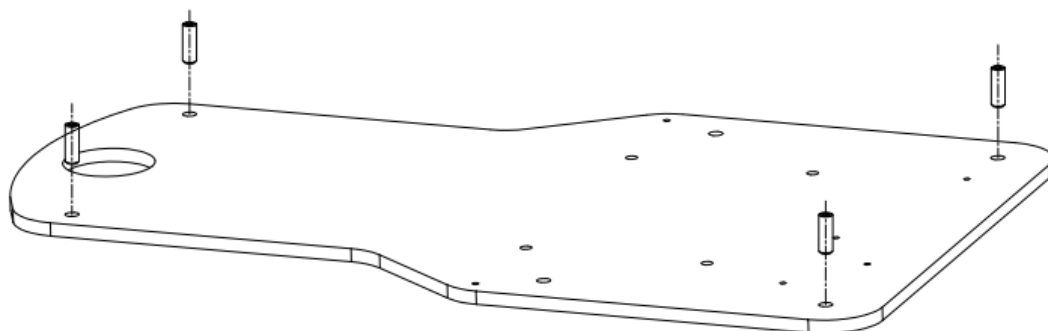


5.2 _ Installing of the Backrest

Figure 8 _ Preparation of the position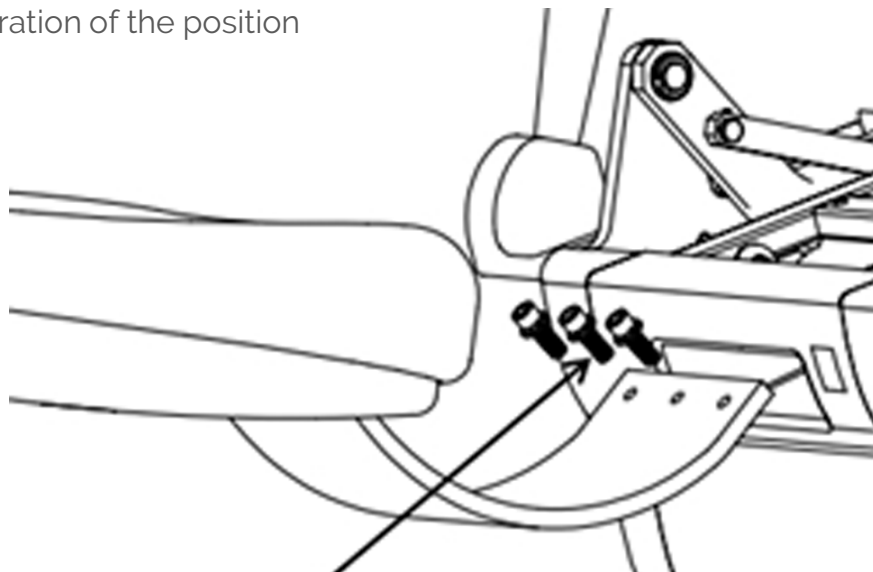

Align the holes of the plate which bears the backrest with the ones of the seat, clench the 3 screws using the hexagonal key.


Figure 9 _ Inserting the fastening screws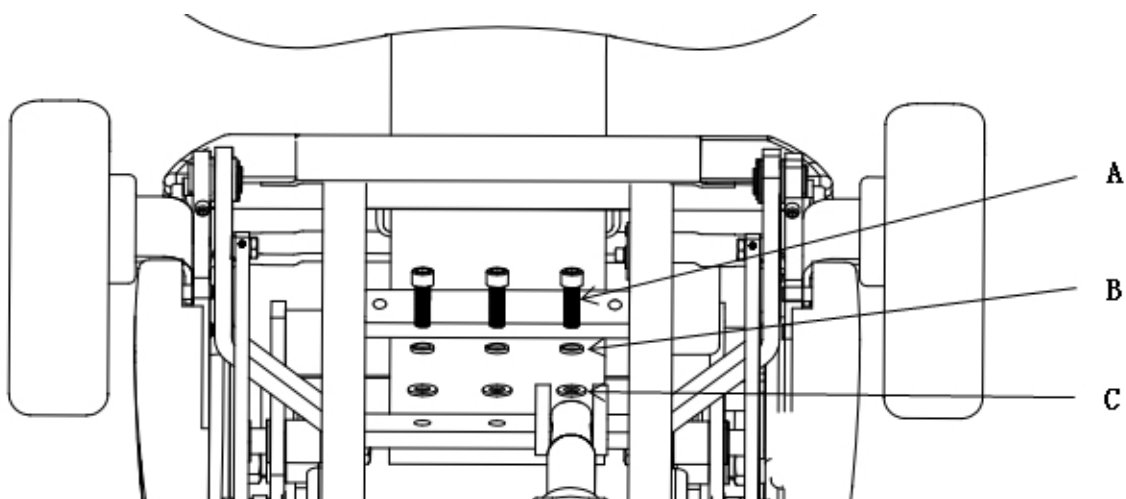

- A. Screw M10*30
- B. Hole of the backrest bearing plate $\Phi 10$
- C. Hole of connection to the seat $\Phi 10$

5.3 Installing of the operating lamp

Figure 10 _ Assembling of the lamp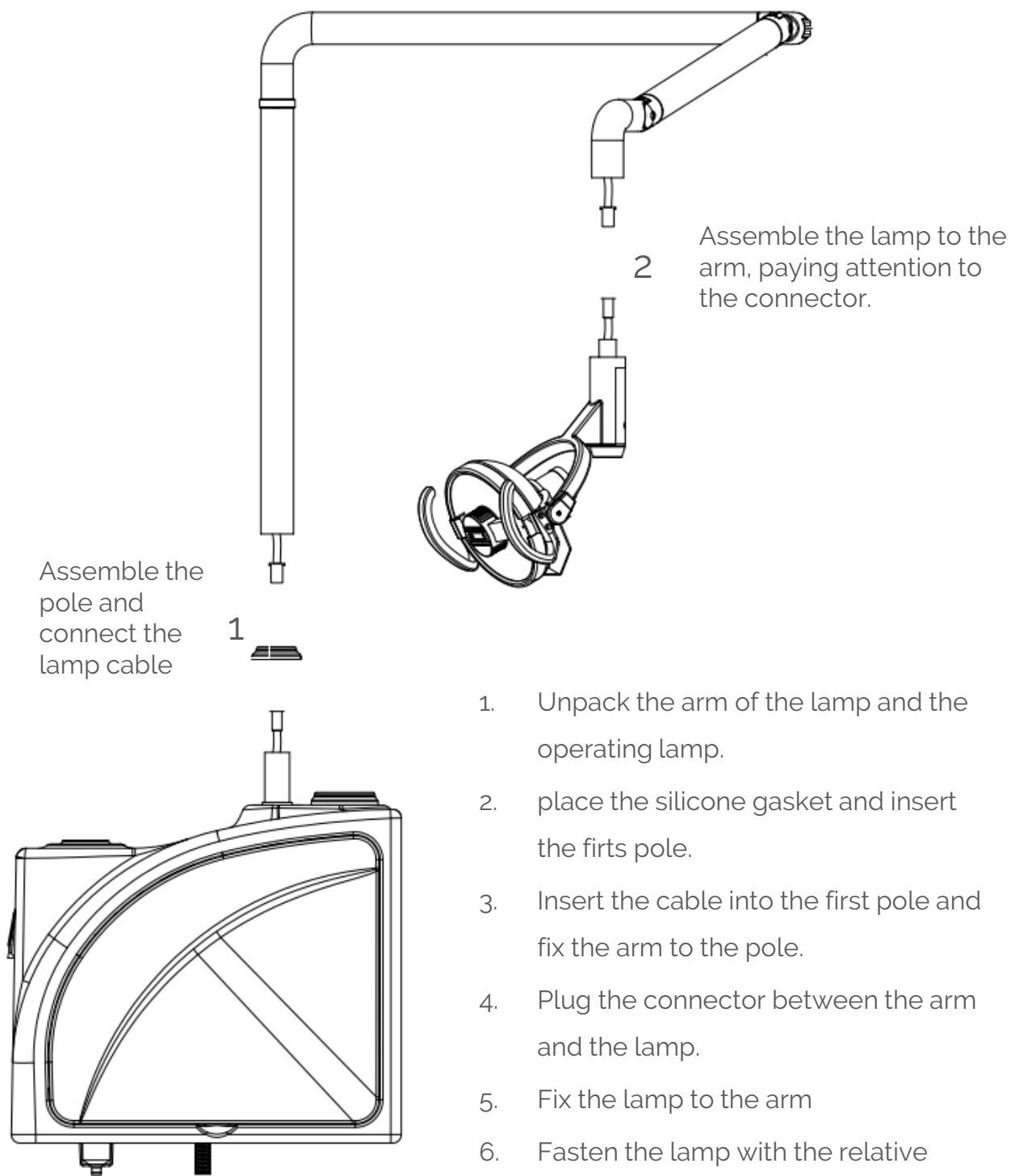


5.4 _ Installing of the Basin

Insert the basin into the relative support placed onto the water unit. Pay attention to properly thread the water and the drain pipes into the suitable space.

Connect the water supply tube for the glass filling to the electrovalve of the water unit. Do the same thing with the water hose of the basin. Connect the drain pipe.

Insert the spouts for the glass filling and the cuspidor rinse.

Figure 11 _ Inserting of the spouts

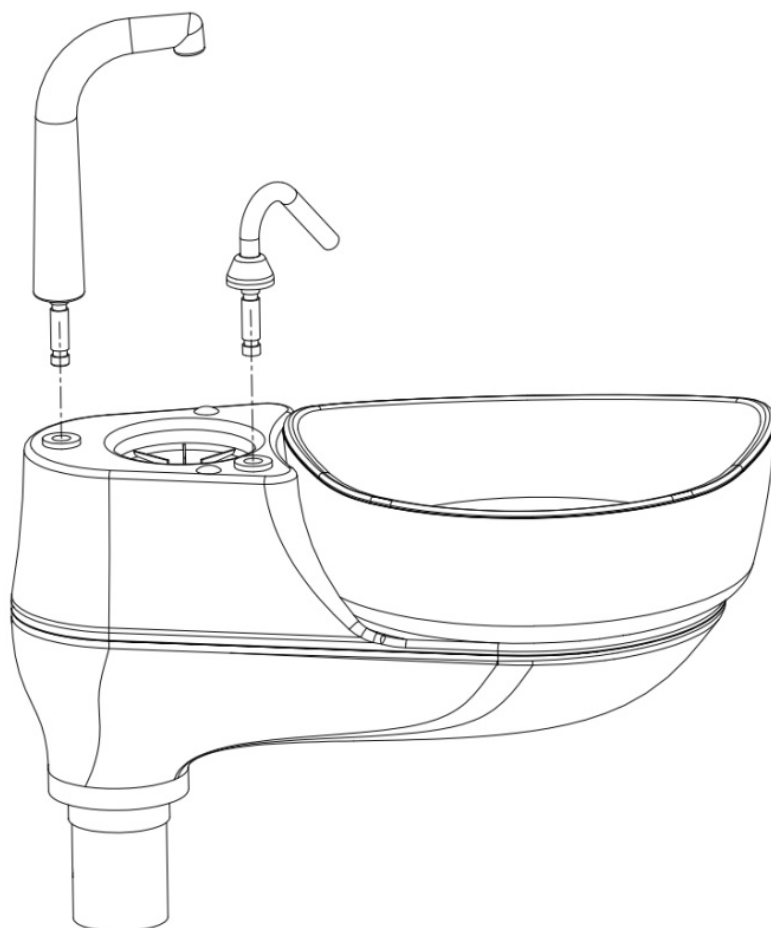6 _ Tests and Adjustments

The specialized technician, trained by SIMPLE&SMART, is required to explain the operation as well as to give information on the correct sanitization procedure to the customer.

After installing SS_ONE/SS_KISS, please verify the proper functioning of the unit:

- Check the movement of the chair and the backrest from the different control units. Check the functioning of the memorized positions.
- Check the correct operation of the handpieces, the correct air, water and spray supply. Make a proper calibration.
- Check the correct balance of the chair and the correct mobility of the arms and the clutches.
- Check the correct functioning of the operating lamp and its intensity.
- Test the correct operation of the pedal controller.
- Check the drains and any possible leak.
- Ensure the proper operation of the glass filling and the cuspidor rinsing.
- Check the correct inlet air and water pressure.
- Check the proper operation of the compression system.

7 _ Serial Numbers

The equipment is provided with a serial number which allows the traceability of the device itself. In the interest of greater supervision all the electronic components (electrical boards, motors, transformer, operating lamp, electrovalves and handpieces) have a serial number as well. In case of repair, the technician must communicate the serial number of the part to be replaced, as well as to take care of return the defective piece to SIMPLE & SMART.

8 _ Classifications and Reference Regulation

Classification: **MEDICAL DEVICES**

Classification of the dental unit according to the rules indicated in Annex IX of Directive 93/42/CE, as amended: Class: **IIA**.

Classification: **ELECTROMEDICAL DEVICES**

Classification of the equipment according to Regulation EN 60601-1 for safety of medical equipment: **Class I - Type B**.

Reference standards: The SS_ONE units are equipments designed in accordance with Regulations CEI EN 60601-1, CEI EN 60601-1-2, EN ISO 13485:2012, EN ISO 13485:2012/AC:2012.

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

SCHEMI TECNICI
TECHNICAL DIAGRAMS
DIAGRAMAS TECNICOS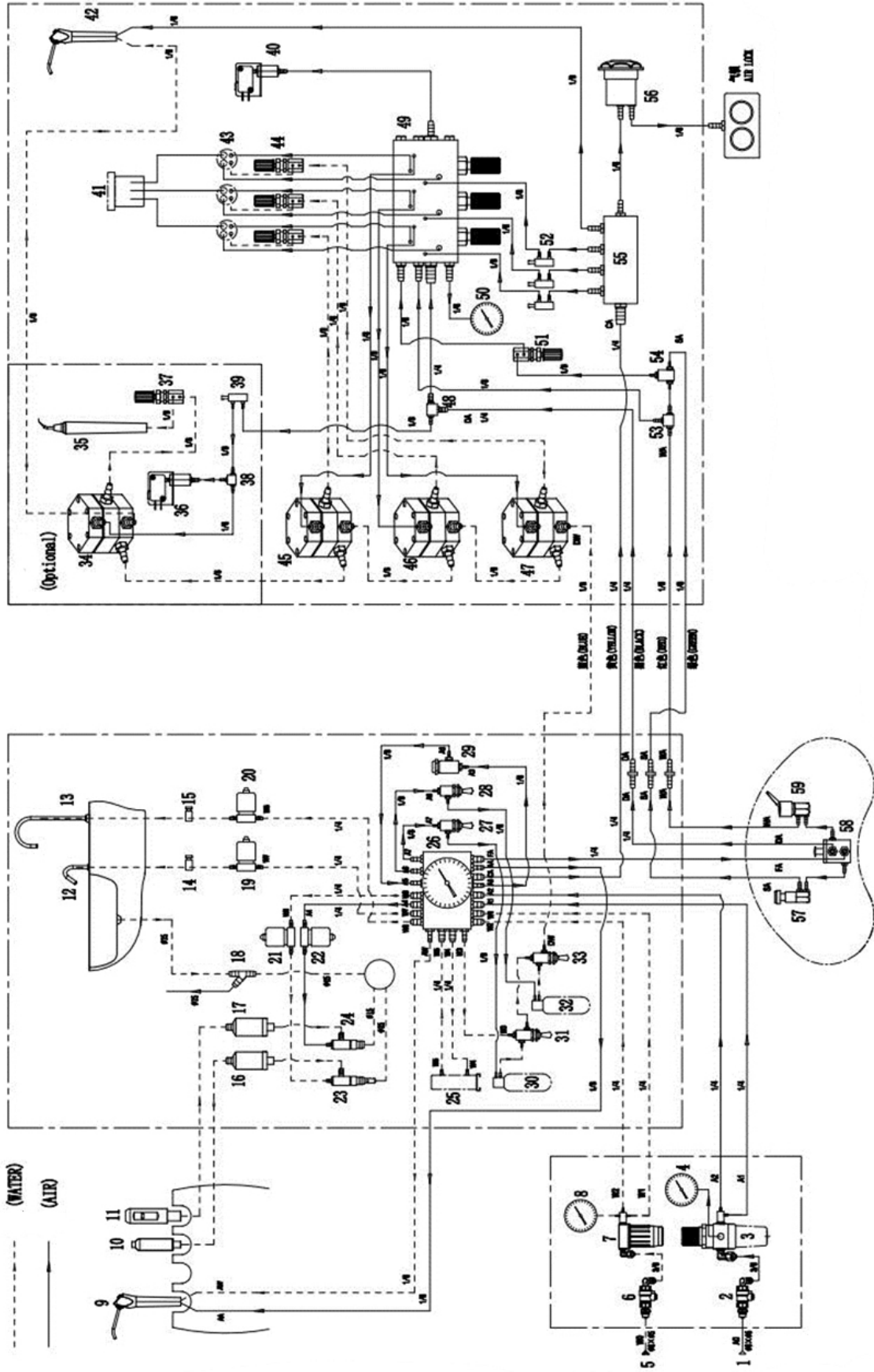

DESCRIPTION

Item	Description	Item	Description
1	Air source	31	Switch water source valve
2	Ball valve	32	Disinfection water bottle
3	Air relief and filter valve	33	Switch water source valve
4	Air pressure gauge	34	Water controlled valve
5	Water Source	35	Scaler
6	Ball valve	36	Air controlled electrical switch
7	Water filter	37	φ2.4-φ2.4 flow adjustment valve
8	Pressure Gauge of water way	38	3-ways connector
9	3-ways syringe	39	Air switch valve(off)
10	Weak suction handle	40	Air controlled electrical switch
11	Strong suction handle	41	Oil collector
12	Rinsing tube	42	3-ways syringe
13	Water supply tube	43	Handpiece
14	Pinch valve	44	φ2.4-φ2.7 flow adjustment valve
15	Pinch valve	45	Water controlled valve
16	Weak suction filter assembly	46	Water controlled valve
17	Strong suction filter assembly	47	Water controlled valve
18	3-ways draining connector	48	3-ways connector
19	Solenoid valve	49	Integrated Valve
20	Solenoid valve	50	Pressure gauge
21	Solenoid valve	51	φ2.4-φ2.4 flow adjustment valve
22	Solenoid valve	52	Air switch valve (on)
23	Weak suction valve	53	3-ways connector
24	Strong suction valve	54	Shuttle valve
25	Water heater	55	6-ways connector
26	Multi ways connector	56	Air switch
27	Air switch	57	Chip blow valve
28	Air switch	58	Foot control valve
29	Air pressure relief valve	59	Wet/Dry Toggle Valve
30	Sel –contained bottle		

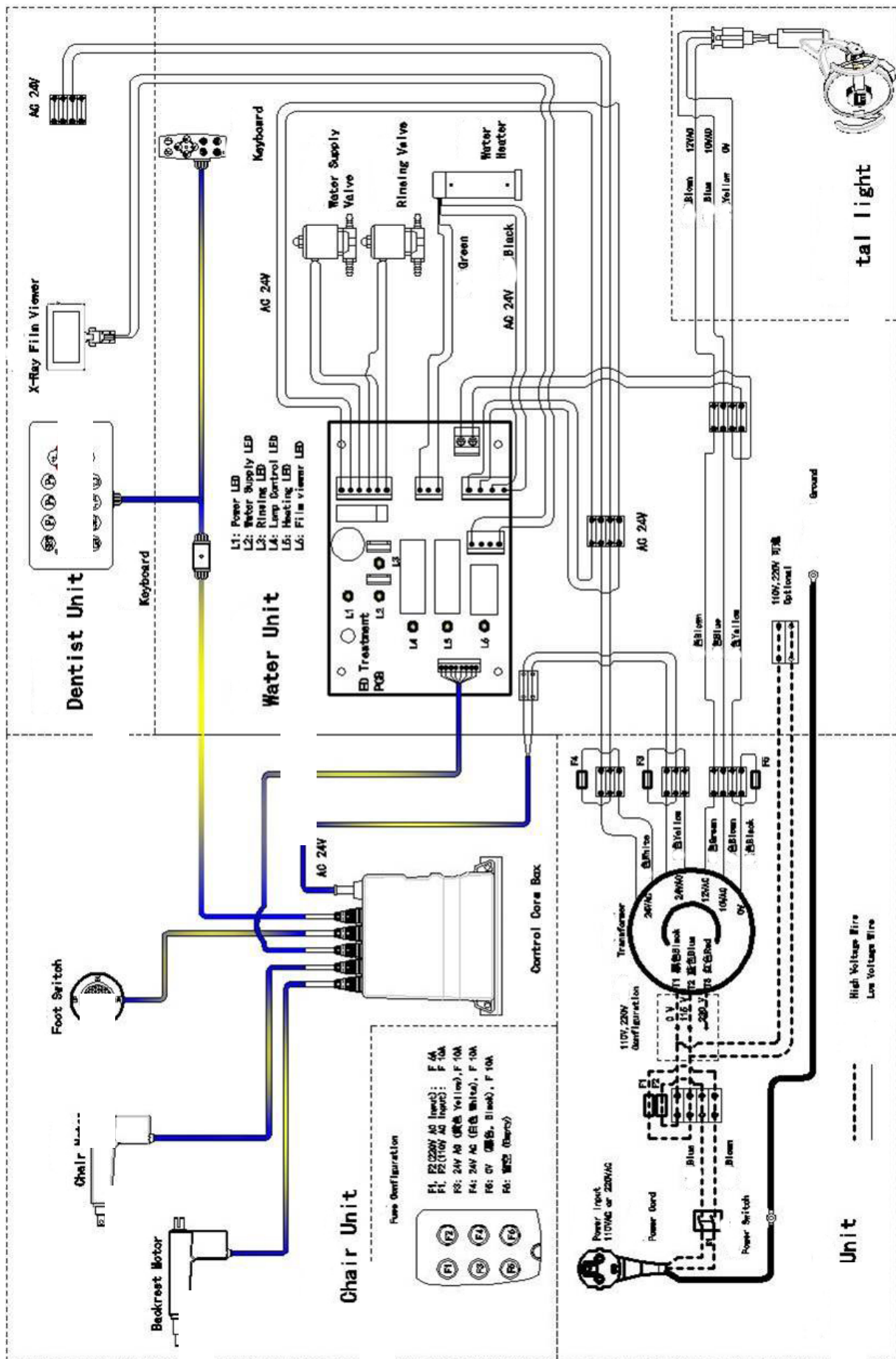CERTIFICATI
CERTIFICATES
CERTIFICADOS

BUREAU VERITAS
Certification

CE Certificate

Issued to

SIMPLE & SMART SRL

Head office and Operative Site:
Via Perola 16, 24021 Albino (BG)- ITALY

Bureau Veritas Italia S.p.A. certifies that the Production Quality Assurance System of the above organization has been audited and found to be in accordance with the requirements of

DIRECTIVE 93/42/EEC as amended

(according to Annex V)

In relation to the following products

Subcategory:	Active dental devices
Generic group:	Dental units
Model:	SS_ONE, SS_KISS
Class:	Ila

Reference BV practice: ZIG. N. 60434385

Certification cycle start date: **03 July 2015**

Subject to the continued satisfactory operation of the organization's Management System, this certificate expires on: **02 July 2018**

Original certification date: **03 July 2015**

Certificate N. **IT261641**

Rev. N 2 of: 28 August 2017

CARMELA DE FEO- Technical Committee Coordinator

This certificate is issued by Bureau Veritas Italia S.p.A. Via Miramare, 15 - 20126 Milan, as a notified body for the Directive 93/42/EEC, with identification number 1370

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Declare that the product: **SS_ONE**

Serial Number:
Production date:

Classification:
in accordance with the Annex
conformity assessment

CLASS IIA - REGULATION 5 - REGULATION 9
IX DIRECTIVE 93/42 EEC. E.S.M.I.
ANNEX V DIRECTIVE 93/42 EEC AMENDED BY
DIRECTIVE 07/47 EC

**is accordant to the essential requirement of D.Lgs. 46/1997 about the Medical Devices
Directive 93/42/CEE e s.m.i. including the directive 2007/47/CE of the D.lgs. 37/2010**

Ref CE certificate n°: IT261641
Released by: BOREAU VERITAS ITALIA S.P.A. - Via Miramare, 15 - 20126 MILANO
Identification number: CE 1370
Original Certification Date: 07-03-2015

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A handwritten signature in blue ink, appearing to read 'G. Veritas', is written over the contact information for Simple&Smart SRL.

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A handwritten signature in blue ink, appearing to read 'G. V. Neri'.

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WARRANTY

The SIMPLE & SMART equipment are guaranteed against defects in material and workmanship for a period of 12 months from the time of installation. Any warranty extensions are allowed only if explicitly written in the contract of sale or invoice. The warranty is on the dental unit, that is the dental chair excluding: handpieces, electric micromotor, Endoscopy System (camera and monitor). The warranty covers only those parts which are not subject to damages due to the use. Any costs related to shipping spare parts, even if under warranty, remain charged to the customer. The complaint of the defects on the equipment must be done within 15 days from installation.

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CONTROL CERTIFICATION

SIMPLE & SMART srl certifies that the dental units SS_ONE and SS_KISS, in addition to the testing carried out during manufacturing, were subject to a further quality control at the logistics center, following a strict quality control process, AS APPROVED BY THE CERTIFICATE OF CONFORMITY OF THE MACHINE. This certificate can be requested at your discretion.

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ELECTRICAL TEST

Electrical inspections must be carried by specialized technician and are borne by the local distributor of SIMPLE & SMART products. Electrical tests must be carried out periodically, according to the current regulations and must be made with a special tested and certified tool. SIMPLE & SMART disclaims any liability if the tests are not carried out, or if are not compliant to the the legislation.

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ASSISTANCE

The assistance service must be guaranteed by the dealer from whom you purchased the equipment, if the machine is not purchased directly from the manufacturer. However, to receive a more complete support service, you can also contact the service center of SIMPLE & SMART by email: assistance@simplesmart.it or by telephone. More details on the service can be found directly on the site www.simplesmart.it

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INSTALLATION SHEET

In the following pages, you will find the form to fill in concerning the correct installation. The enabled technician is required to fill it after installation. The form must be signed by the technician and the customer and sent electronically to one of the simple & smart mail addresses. A copy of the form must be kept by the customer. The correct installation form is necessary in order to obtain warranty service.

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Herewith, the above mentioned installer, declares to have installed and tested the machine correctly, in accordance with the procedures and technical parameters of the device itself. The customer also declares to be aware of how to use the dental unit, of the conditions contained in the following warranty and to have read the manual.

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FIRMA/Signature/Firma
INSTALLATORE/Installer/Instalador

FIRMA/Signature/Firma
CLIENTE/Customer/Cliente

Simple&Smart Srl

Via Perola 16
24021, Albino, Bergamo.

ITALIA

www.simplesmart.it