

**Directions for use**

# Prisma<sup>®</sup>-Gloss<sup>™</sup><sub>y</sub>

# Prisma<sup>®</sup>-Gloss<sup>™</sup> Extrafine

## Composite Polishing Paste

### INDICATIONS

Prisma-Gloss Paste is designed for final polishing of composite restorations or veneers.

**Caution:** U.S. Federal law restricts this device to sale by or on the order of a dentist. For dental use only.

### COMPOSITION

Prisma<sup>®</sup>-Gloss<sup>™</sup> and Prisma<sup>®</sup>-Gloss<sup>™</sup> Extrafine are single component, water dispensed, polishing pastes. These pastes will impart improved smoothness and surface luster to anterior and posterior composite restorations.

### PROCEDURE

1. Proceed with normal procedures for gross reduction and finishing of excess cured composite.
2. Apply a small amount of Prisma<sup>®</sup>-Gloss<sup>™</sup> material to the surface of the polishing cup. Work the surface of the restoration, dry initially at moderate speed and pressure.
3. To increase surface luster, ADD WATER IN SMALL AMOUNTS to dilute paste, then use a light circular buffing action. Repeat as needed to produce a smooth surface.
4. Rinse the Prisma<sup>®</sup>-Gloss<sup>™</sup> from the tooth surface. THIS IS THE FINAL STEP FOR MICROFILLED COMPOSITES. Continue with the next step for hybrid composite materials.
5. Apply Prisma<sup>®</sup>-Gloss<sup>™</sup> Extrafine paste to the polishing cup as described above in steps 3 & 4 and lightly buff the surfaces for the final luster. A high luster should rapidly develop in 20-30 seconds of polishing. Rinse all surfaces and discard the polishing cup after use.

### Caution:

Do not use Prisma<sup>®</sup>-Gloss<sup>™</sup> Extrafine on microfilled composites.

©1990 DENTSPLY International Inc. All Rights Reserved  
DENTSPLY Caulk  
DENTSPLY International Inc.  
Milford, DE 19963-0359

Authorized EU Representative  
DENTSPLY DeTrey GmbH  
D-78467 Konstanz  
Germany

Swiss Representative  
DENTSPLY DeTrey GmbH, Konstanz  
Zweigniederlassung Baar  
CH 6342 Baar

Printed in U.S.A.

531400 (R-12/98)