NAKANISHI INC. 

www.nsk-inc.com

700 Shimohinata Kanuma-shi
Tochigi 322-8666,
Japan

NSK United Kingdom Ltd

www.nsk-uk.com

Office 5, Gateway1000,
Arlington Business Park, Whittle Way,
Stevenage, SG1 2FP, UK

NSK America Corp

www.nsk-inc.com

700 Cooper Court
Schaumburg, IL 60173,
USA

Specifications are subject to change without notice.

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NSK Middle East

www.nsk-inc.com

Room 6EA-701, 7th Floor, East Wing No.6
Dubai Airport Free Zone,
PO Box 54316 Dubai, UAE

NSK France SAS

www.nsk.fr

19 avenue de Villiers
75017 Paris,
France

NSK Oceania Pty Ltd

www.nsk-inc.com

Unit 22, 198-222 Young St.
Waterloo, Sydney,
NSW 2017, Australia

'09.03.30 

OPERATION MANUAL

NSK

 0197

LED Coupling

PTL-CL-LED/PTL-CL-LED III KCL-LED

Please read this Operation Manual carefully
before use, and file for future reference.

MADE IN JAPAN

Powerful Partners®

**CAUTION**

- When operating the Product always consider the safety of the patient.
- The product is designed only for clinical dental use by qualified personnel.
- Do not attempt to disassemble the product or tamper with the mechanism.
- Should the Product function abnormally, cease operation immediately and return the Product to an authorised dealer for repair.
- Do not allow any impact on to the Product. Do not drop the Product.
- Do not connect or disconnect the hose or handpiece until the drive motor has completely stopped.
- Do not wipe with, or clean or immerse in, high acid water or sterilizing solutions.
- Confirming the operation manual of the hose to connect and handpiece.
- Do not attempt hydrothermal cleaning or rinse it in water. It may cause malfunction or electric shock.
- Follow the instrument manufacturer's instructions.
- Ensure clean, dry air supply. Moisture and debris in air supply may cause the motor to malfunction.

1. Features

- There is no generation of heat in the light source, power saving and long life LED.
- It can adjust the amount of water by water adjusting ring. (PTL-CL-LED-III, KCL-LED)
- Power supply for illumination is regardless of polarity. You can use variety of unit.
- Non-Retraction Valve which prevents the water flow back of water is built in.

2. Setting

CAUTION

- Do not connect or disconnect the hose or handpiece until air pressure has completely stopped.
- Follow the instrument manufacturer's instructions.
- This coupling can connect with a hose with light and ISO Standard 4 hole.(Fig.1) If it connects with a hose without light function, the light terminal will deform and it will become a cause of failure.

Fig. 1

(1) Connecting

- 1) To connect, align and insert carefully and firmly the pin of the coupling into the pin holes of the hose, and fasten the hose nut securely.(Fig.2)
 - * Coupling cap as attached.(PTL-CL-LED/PTL-CL-LED III)
 - * Wrench (KCL-LED)
- 2) Insert handpiece into coupling straight until it click.

CAUTION

Ensure the handpiece is attached tightly to the coupling.

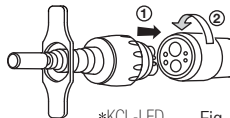
Fig. 2

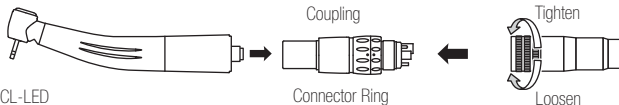
Fig. 3

(2) Disconnecting

- 1) To disconnect, pull back the connector ring and gently pull out.
- 2) Loosen the hose nut and gently pull out the hose.

CAUTION

Ensure the handpiece is attached tightly and properly to the coupling. DO not pull the connector ring during air pressure supply. If the connector ring pulled during air pressure supply, handpiece may fly away.


3. Controlling Water Supply (PTL-CL-LED III, KCL-LED)

Water supply is controlled by turning the Water Adjustment Ring.

PTL-CL-LED III Water Adjustment Ring can rotate both sides. Flooding decreases when it turns to back side or front side start to mark keep the mark. When the Water Adjustment Ring is turned 180 degree, water supply stop. (Refer to Fig.4 PTL-CL-LED III)

KCL-LED Water Adjustment Ring can rotate back side to decrease the water supply. (Refer to Fig.3 KCL-LED)

PTL-CL-LED III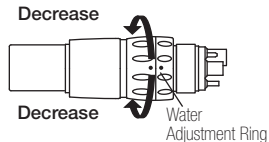

KCL-LED

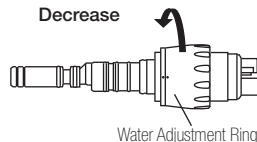
Fig. 4 5

4. Maintenance

After each use, clean the surface of the coupling with brush etc. (Do not use metal brush), and wipe it with the cotton moustened with disinfecting alcohol.

 This coupling can not be Autoclave sterilization.

 This product can not be cleaned and disinfected with a Thermo-Disinfector.

CAUTION

- Do not wipe with, or clean or immerse in, high acid water or sterilizing solutions.
- Do not connect or disconnect the hose or handpiece until the drive motor has completely stopped.

5. Non-Retraction Valve

A Non-Retraction Valve is equipped in the Coupling Joint, which shuts off the water retraction right at the handpiece head, to prevent oral fluids sucked into the water line. (Fig. 5)
To replace the valve, remove the back-end gasket. Pull and remove the water tube, and replace the Non-Retraction Valve.

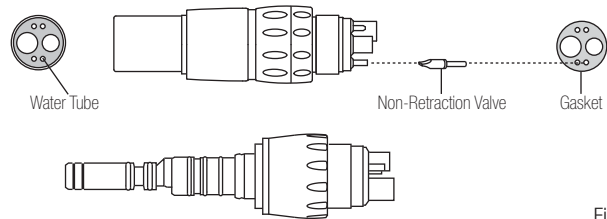
Fig. 5

6. Changing O-Ring (KCL-LED)

CAUTION

- Maintenance the coupling after the air supply is stopped completely.
- Do not pull the O-ring with excessive power.

(1) Changing O-Ring


- 1) Remove the worn O-rings.
- 2) Insert new O-rings the O-ring grooves.

Optional O-rings (2 pcs.)

: Order No. 0318-040100

Optional O-rings (3 pcs.)

: Order No. 0318-068080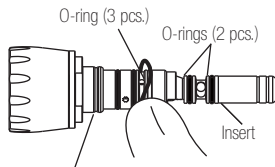

* Picture shows KCL-LED

Fig. 6 8

7. Coupling Cap (PTL-CL-LED/PTL-CL-LED III)

(1) Connecting

To connect, align and insert carefully and firmly the pin of the coupling cap into the pin holes of the coupling, and turn the ring until you hear the noise "Click".

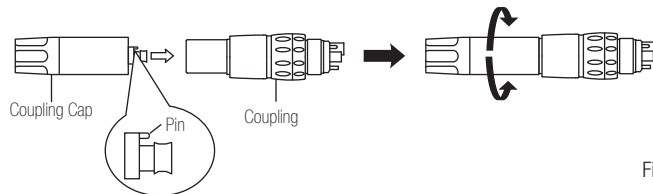
Fig. 7


2) Approach in the use of coupling cap

As instruction wrench

When coupling is connecting to a hose, use it as safeguard use of the coupling.

As for protection against dust

During the sterilization, when coupling is removed from handpiece, the inside of coupling be exposed and garbage, various germs, should enter, It is not sanitarly good, therefore use it as a protection-against-dust cap.

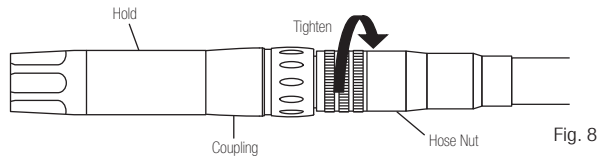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*KaVo® MULTiflex® is copyright for Kaltenbach & Voigt GmbH & Co.(Germany).
*KaVo® MULTiflex® ist ein eingetragenes Warenzeichen der Kaltenbach & Voigt GmbH & Co.
*KaVo® MULTiflex® est une marque déposée de Kaltenbach & Voigt GmbH & Co. (Allemagne).
*KaVo® MULTiflex® es una marca registrada de Kaltenbach & Voigt GmbH & Co (Alemania).