roeko Stay-put

NON-IMPREGNATED

| Instructions for use | EN |
|----------------------|----|
|----------------------|----|

Use:

A product for improving the results of dental impres-sions.

Application:

Cut *Stay-put* to the required length. *Stay-put* can be soaked in GINGIVA LIQUID or another gingival retraction solution of your choice.

Please note: Don't use GINGIVA LIQUID in cases of necrotizing Gingivitis ulcerosa. Please follow instructions for use of the chosen liquid. GINGIVA LIQUID is not available in all countries.

Important:

For best results when using *Stay-put* please follow instructions carefully.

Place the centre of the cord into the sulcus. Next work out towards the ends of the cord.

Overlapping is not necessary.

Leave the non-impregnated cord for at least 5 minutes in the sulcus. When using GINGIVA LIQUID allow 1-3 minutes for the liquid to take effect. Remove the cord directly before taking the impression.

Compositions:

Stay-put is a unique combination of softly braided retraction cord and an ultra fine copper filament bounded with nylon.

| | | |
|---|--------|-------------|
| 0 | x-fine | REF 521 000 |
| 1 | fine | REF 521 001 |
| 2 | medium | REF 521 002 |
| 3 | thick | REF 521 003 |

| |
|---------------------------|
| Contraindications: |
| None |

| |
|---|
| Further advise: |
| Avoid touching the cord with electrotomy and laser equipment. |

| |
|--|
| Warning: |
| Medical devices should be kept out of reach of children. |

| |
|--|
| Use and storage: |
| <ul style="list-style-type: none">for single use only for dental use only the lid has to be closed immediately after use and to be stored in a dry place shelf life 5 years this medical device should not be used after the expiration date |

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