200 mm

262 mm

C-RAM BOOSTER

PRIMER FOR
All ceramic surfaces

C-RAM BOOSTER

FR

DESCRIPTION

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EN

DESCRIPTION

C-Ram Booster is a surface conditioner for ceramic materials such as zirconia, porcelain, glass ceramics and metal ceramics.. This primer dramatically improves adhesion between ceramic surfaces and resin materials. It does not require additional conditioning such as sandblasting. Its low viscosity and excellent wetting properties promote ideal surface coverage.

INDICATIONS

- Preparation of ceramic crowns, veneers and inlays prior to cementation
- Preparation of fractured ceramic restorations for repair with resin materials

INSTRUCTIONS FOR USE

1. Clean ceramic surface with alcohol or acetone and dry well with oil-free air. 2. Open C-Ram Booster bottle and dispense 1-2 drops into mixing well, replace bottle cap immediately. 3. Apply generous amount of primer onto ceramic surfaces using a brush or micro brush. 4. Wait 1 minute. 5. Dry for 20s with oil-free air. 6. Continue with clinical procedure.

ESSENTIAL INGREDIENTS

Ethanol, water, silane, Multi-Functional Monomers

CAUTIONS

- Shelf life is 2 years after the date of manufacturing.
- Avoid contact with skin, oral soft tissues and eyes.
- If accidental contact occurs, rinse with copious amount of water and seek medical attention immediately.

STORAGE

- Do not store above 24°C (75°F). • For optimum freshness, keep refrigerated. • Do not freeze. • Tightly recap after use. • Do not use after expiration date.

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