## Chewies™ Aligner Tray Seater

DIRECTIONS FOR USE  
INSTRUCCIONES DE USO  
INSTRUÇÕES DE UTILIZAÇÃO  
MODE D'EMPLOI  
ISTRUZIONI PER L'USO  
GEBRAUCHSANWEISUNG  
GEBRUIKSAANWIJZING

Manufacturer:  
**Dentsply Sirona Orthodontics Inc.**  
7290 26th Court East  
Sarasota, FL 34243 USA

www.dentsplysirona.com/orthodontics-e-labeling

SINGLE  
PATIENT  
USE

EC REP

Emergo Europe  
Prinsessegracht 20  
2514 AP, The Hague  
The Netherlands

### Chewies™ Aligner Tray Seater

ENG

#### INTENDED USE:

Intended to provide aid for fitting the aligner/retainer to patient teeth by chewing for short periods of time while placing the tray on the teeth under normal oral body temperature ranges.

#### INDICATIONS FOR USE:

Indicated for patients wearing thermoformed aligners or retainers.

#### PERFORMANCE CHARACTERISTICS AND CLINICAL BENEFITS:

This device is made of a soft, chewable material to aid the patient in properly seating their aligner or retainer.

#### CONTRAINDICATIONS:

None known.

#### WARNINGS:

Single patient use only.

Use of excessive pressure to seat the aligner or retainer may cause damage to it.

#### PRECAUTIONS:

For momentary use for seating aligner or retainer only.

Seat only one aligner or retainer at a time. Trying to seat two at once may result in damage to the aligners or retainers.

Do not reuse the product on another person. Risks of reuse include infection and cross-contamination.

Report serious incidents that have occurred in relation to the device to your healthcare professional or the device manufacturer and/or the Competent Authority in Member State of residence, or other applicable regulatory agency.

#### ADVERSE REACTIONS:

None known.

#### STEP BY STEP INSTRUCTIONS:

1. Place the aligner or retainer over the teeth by hand and apply slight pressure until it is seated most of the way.
2. Insert the Chewies™ Aligner Tray Seater into your mouth and position it between the upper and lower arches.
3. Gently bite down on the Chewies™ Aligner Tray Seater and work the aligner or retainer into place until fully seated.

#### SAFE DISPOSAL OF DEVICE:

Dispose of in the regular household trash, as you would your toothbrush or used dental floss.

#### WARRANTY AND LIMITED REMEDY

Manufacturer warrants that this product is free from defects in materials and manufacture.

Manufacturer's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to replacement merchandise or refund of the purchase price.

All implied warranties, including implied warranties of merchantability or fitness for any particular purpose or use are disclaimed.

#### LIMITATION OF LIABILITY

Except where prohibited by law, Manufacturer will not be liable for any loss or damages arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. This limitation does not apply to third party personal injury claims.

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